

# AeroEclipse™ II

## Elbow Adapter for Breath Actuated Nebulizer (BAN)

### Instructions for Use

1. Remove the mouthpiece from side opening of the nebulizer.
2. Insert the Elbow Adapter into the mouthpiece opening, until it is secure.
3. Attach any standard aerosol mask to the Elbow Adapter, making sure all connections are secure.
4. Follow the user instructions enclosed with the **AeroEclipse®** II BAN, and use as directed.

### Cleaning Instructions

Disengage Elbow Adapter from **AeroEclipse®** II BAN and wash per nebulizer instructions in a warm water/liquid dish detergent solution. Rinse in warm water. Ensure parts are dry before reassembling.

### Single Patient Use

The device should be replaced after approximately 7 days or immediately if components are cracked or the nebulizer does not function properly. This device is not intended to be reprocessed and should not be autoclaved or boiled.

LATEX FREE

Manufactured for:

**Monaghan Medical Corp.**

5 Latour Ave., Suite 1600  
Plattsburgh, NY 12901 USA  
Tel. 800-833-9653 Fax 518-561-5088

Packaged by:

Trudell Medical International

725 Third Street  
London, Ontario, Canada

#### REORDER NUMBERS

<b>AeroEclipse®</b> II BAN	64594050
<b>AeroEclipse®</b> II BAN w/ elbow	65050E
Elbow Adapter	65005
Small Mask	65750
Large Mask	65850

