


## DISPOSABLE AEROECLIPSE® AEROSOL MASK ASSEMBLY

Intended for use with the Monaghan **AeroEclipse®** Breath Actuated Nebulizer (BAN)

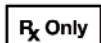

**MEDIUM** PN 65950- AE Mask, Medium, DISPOSABLE, cs/50

LATEX-FREE

### INSTRUCTIONS FOR USE:

1. Remove the mask assembly from its packaging, inspect for damaged components and verify the swivel elbow is correctly secured to the mask.
2. Remove the mouthpiece from the **AeroEclipse®** BAN.
3. Attach the mask assembly to the **AeroEclipse®** BAN.
4. Follow the operating instructions supplied with the **AeroEclipse®** BAN.
5. Ensure the mask maintains a good face seal prior to operating the **AeroEclipse®** BAN. The built in mask elbow allows for rotational positioning.
6. If you are unable to trigger the **AeroEclipse®** BAN, it can be manually activated by pressing/releasing the green feedback button in coordination with inspiration/expiration, or by operating the device in continuous mode (see operating instructions supplied with the **AeroEclipse®** BAN).

### CAUTION:

For single patient use only.

Store in a cool dry place.

A clinician should always be present during the treatment.



**monaghan™**

Manufactured for:  
Monaghan Medical Corporation  
5 Latour Ave., Suite 1600  
Plattsburgh, NY 12901  
1.800.833.9653  
[www.monaghanmed.com](http://www.monaghanmed.com)

By: Flexicare Medical Limited • Cynon Valley  
Business Park • Mountain Ash, CF45 4ER • U.K.  
T/+44(0)1443 474647  
ORIGIN- Made in China

© 2009 Monaghan Medical Corp.

PN 65809-00 07/09

AeroEclipse® BAN is a registered trademark of  
Trudell Medical International.

Venticaire® is a registered trademark of Flexicare Medical Limited.

