


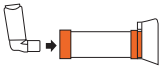
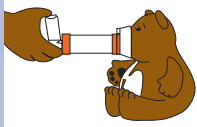
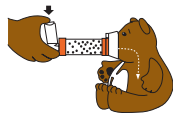
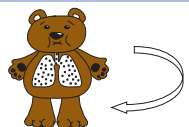
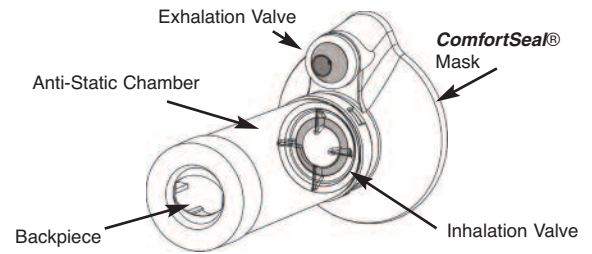
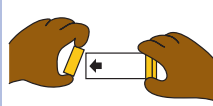

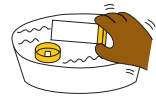
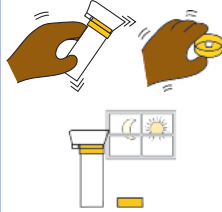
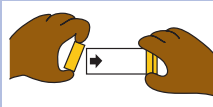


Directions for Use (Single Patient Use Only)

ENGLISH	
<p>Product Reorder Number Part Number:</p> <p>Small aVHC 88710 cs/10 Medium aVHC 78710 cs/10</p>	<p>This product can be used right out of package. Before use, ensure these instructions and the instructions supplied with the Metered Dose Inhaler (MDI) have been read.</p>
<p>1</p>  <p>SMALL MASK MEDIUM MASK</p>	<p>Carefully examine the product for damage, missing parts or foreign objects. Remove any foreign objects prior to use. The product should be replaced IMMEDIATELY if there are any damaged or missing parts. If necessary, use the Metered Dose Inhaler (MDI) alone until a replacement is obtained. If the patient's symptoms worsen, please seek immediate medical attention.</p>
<p>2</p> 	<p>Remove cap from the MDI.</p>
<p>3</p> 	<p>Shake the MDI immediately before each use as per the instructions supplied with the MDI.</p>
<p>4</p> 	<p>Insert the MDI into the backpiece of the chamber.</p>
<p>5</p> 	<p>Apply mask to face and ensure an effective seal.</p>
<p>6</p> 	<p>Breathe out gently and depress the MDI at the beginning of a slow inhalation. Maintain seal for 5-6 breaths after the MDI is depressed. Administer one (1) puff at a time.</p>
<p>7</p> 	<p>Follow instructions supplied with the MDI on how long to wait before repeating steps 3-6 as prescribed.</p>



Cleaning Instructions

ENGLISH	
<p>1</p> 	<p>This product can be used right out of the package and then cleaned weekly.</p> <p>Remove the Backpiece only. Do not tamper with valve during cleaning or disassemble the product beyond what is recommended or damage may result.</p>
<p>2</p> 	<p>Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently.</p>
<p>3</p> 	<p>Rinse parts in clean water.</p>
<p>4</p> 	<p>Shake out excess water from the parts and allow to air dry in a vertical position. Ensure parts are dry before reassembly.</p>
<p>5</p> 	<p>To reassemble, press firmly to attach the Backpiece.</p>

AeroChamber
Plus® ZSTAT®
anti-static Valved Holding Chamber with Small or Medium ComfortSeal® Mask



monaghan™

Monaghan Medical Corporation
5 Latour Ave., Suite 1600 Plattsburgh, NY 12901
For additional information visit www.monaghanmed.com

© 2005 Monaghan Medical Corporation
™ and ® are trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. This device was Made in the USA from Canadian and US parts.

Cautions

- To ensure proper performance this product should only be cleaned according to these instructions. DO NOT clean in dishwasher.
- Do not leave the chamber unattended with children.
- Federal (USA) law restricts the sale of this device on or by the order of a physician.

Rx Only

Notes

- Storage and operating range 5° C - 40° C (41° F - 104° F) at 15 to 95% relative humidity.
- Product may need to be replaced after 12 months of use. Environmental conditions, storage and proper cleaning can affect product life span.
- THIS PRODUCT CONTAINS NO LATEX.
- Do not share this medical device.
- Clarity of the chamber is a result of the properties of the anti-static material.
- aVHC = anti-static Valved Holding Chamber

Indications for Use: This product is intended to be used by patients who are under the care or treatment of a physician or licensed health care professional. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers. The intended environments for use include the home, hospitals and clinics.