AeroChamber Plus Z STAT™

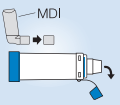
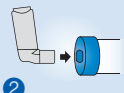
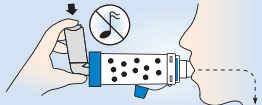
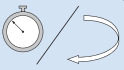
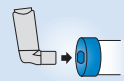
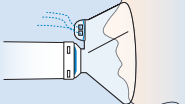
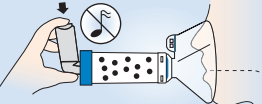
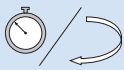
Anti-Static Valved Holding Chamber

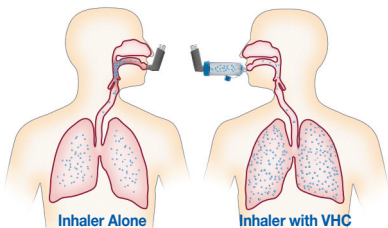
Below, please find abbreviated instructions, but remember to read the complete instructions that are packaged with your **AeroChamber Plus® Z STAT®** anti-static Valved Holding Chamber (aVHC) device.

MOUTHPIECE

LARGE MASK

<p>1</p>  <p>Remove cap from the MDI and chamber. <i>Before use, ensure the instructions supplied with the MDI and chamber have been read.</i></p>	<p>2</p>  <p>Shake the MDI immediately before each use. Insert the MDI into the backpiece of the chamber.</p>	<p>3</p>  <p>Put mouthpiece into mouth and close lips around it to ensure effective seal.</p> <p>Apply mask to face and ensure an effective seal.</p>	<p>4</p>  <p>Breathe out gently and depress the MDI at the beginning of a slow inhalation. Breathe in slowly and deeply through the mouth until a full breath has been taken. Administer one (1) puff at a time. Slow down inhalation if you hear the FLOWSignal® Whistle sound.</p> <p>Breathe out gently and depress the MDI at the beginning of a slow inhalation. Maintain seal for 5-6 breaths after the MDI is depressed. Administer one (1) puff at a time. Slow down inhalation if you hear the FLOWSignal® Whistle sound.</p>	<p>5</p>  <p>Follow instructions supplied with MDI on how long to wait before repeating instructions as prescribed.</p>
<p>1</p>  <p>Remove cap from the MDI and chamber. <i>Before use, ensure the instructions supplied with the MDI and chamber have been read.</i></p>	<p>2</p>  <p>Shake the MDI immediately before each use. Insert the MDI into the backpiece of the chamber.</p>	<p>3</p>  <p>Apply mask to face and ensure an effective seal.</p>	<p>4</p>  <p>Breathe out gently and depress the MDI at the beginning of a slow inhalation. Breathe in slowly and deeply through the mouth until a full breath has been taken. Administer one (1) puff at a time. Slow down inhalation if you hear the FLOWSignal® Whistle sound.</p> <p>Breathe out gently and depress the MDI at the beginning of a slow inhalation. Maintain seal for 5-6 breaths after the MDI is depressed. Administer one (1) puff at a time. Slow down inhalation if you hear the FLOWSignal® Whistle sound.</p>	<p>5</p>  <p>Follow instructions supplied with MDI on how long to wait before repeating instructions as prescribed.</p>

Manufactured by:

monaghan™

Monaghan Medical Corporation
5 Latour Avenue, Suite 1600
Plattsburgh, NY 12901
www.monaghanmed.com

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© 2010 Monaghan Medical Corporation™ y © son marcas comerciales y marcas comerciales registradas de Monaghan Medical Corporation o de una empresa afiliada de Monaghan Medical Corporation. Estos dispositivos fueron fabricados en los EE.UU. con piezas estadounidenses y canadienses.