

AeroEclipse™

Filtered Mouthpiece for use with **AeroEclipse®** Breath Actuated Nebulizer (BAN)

INSTRUCTIONS FOR USE:

1. Remove the filtered mouthpiece assembly from its packaging and ensure all connections are secure.
2. Replace the mouthpiece of the **AeroEclipse®** BAN with this filtered mouthpiece.



3. Instruct the patient to inhale and exhale through the filtered mouthpiece to verify movement of the green feedback button located at the top of the **AeroEclipse®** BAN.
4. Follow the operating instructions supplied with the **AeroEclipse®** BAN.
5. Inspect the filtered mouthpiece for damaged or missing components prior to each use.

SINGLE PATIENT USE

The filtered mouthpiece should be replaced after approximately 7 days or immediately if damaged or if components are missing. This assembly is not intended to be soaked, rinsed, washed or sterilized. It should be removed before cleaning the **AeroEclipse®** BAN.

CAUTION:

Federal (USA) law restricts the sale of this device on or by the order of a Physician.

Rx Only

NOTES:

This product contains no latex.

Store at normal room temperature, 0° - 30° C (32° - 92°F)

Reorder number: 60050



monaghan™

Monaghan Medical Corporation
5 Latour Ave., Suite 1600
Plattsburgh, NY 12901 USA
1.800.833.9653

www.monaghanmed.com