Below, please find abbreviated instructions, but remember to read the complete instructions that are packaged with your AEROECLIPSE® XL Reusable Breath Actuated Nebulizer (R BAN) device.

**1** The AEROECLIPSE® XL R BAN can be used with a mask by removing the supplied mouthpiece and inserting the mask into the mouthpiece opening.

**2** Untwist and remove the nebulizer top. Carefully place prescribed medication into nebulizer cup. Reattach the nebulizer top; gently hand tighten.

**3** Attach one end of the supplied tubing to the fitting on the bottom of the nebulizer and the other end to your compressor. Be sure both ends of the tubing are secure.

**4** Turn on your compressor while sitting in a comfortable, upright position. Place the mouthpiece in your mouth being sure not to cover the exhalation valve with your lips.

**5** Inhale slowly and deeply. As you breathe in the green button on top of the nebulizer will move into the “down” position.

## AEROECLIPSE® XL REUSABLE BREATH ACTUATED NEBULIZER (R BAN)

**6** Exhale normally. The green button will return to the “up” position when you are not inhaling.

**7** Continue to breathe normally on the nebulizer until it begins to make a “sputter” noise or as instructed by your physician or health care provider.

**8** Turn off the compressor when finished with your treatment. Clean product between each treatment as directed in the owner’s manual.

This product is for single patient use only and has a 6 month life if properly used, cleaned and maintained.

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Las palabras o frases acompañadas por ™ o ® son marcas comerciales y marcas registradas de Monaghan Medical Corporation o de una filial de Monaghan Medical Corporation. © 2009–2016 Monaghan Medical Corporation. † marcas comerciales y marcas registradas de las respectivas empresas.