**It's important to remember the following tips to gain a child's cooperation:**

Never force the mask on the child or attempt to treat while the child is crying.

Give praise and rewards whenever the child is using the chamber correctly.

Practice short "pretend" treatments by having the mask near the child's face.

Remember to read the complete instructions that are packaged with your AEROCHAMBER® anti-static Valved Holding Chamber (aVHC) device.

5 Latour Avenue  
Suite 1600  
Plattsburgh, NY 12901

518.561.7330 **phone**  
800.833.9653 **toll free**  
518.561.5088 **fax**

monaghanmed.com

## AEROCHAMBER® aVHC MASK TIPS

Use teddy bears, dolls or even adults to "pretend" to use the chamber and make it look like a game.

Let the child play with the chamber while offering encouragement.

Give the treatment when the child is happy, playing or sleeping.

Give the child stickers to put on their chamber to make it their own.

Words or phrases accompanied by ™ and ® are trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. © 2015 Monaghan Medical Corporation. † trademarks and registered trademarks of the respective companies.

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