VersaPAP™ Device


Indications for Use

The **VersaPAP™** device is indicated for the treatment and prevention of atelectasis. The **VersaPAP™** device also has the ability to provide supplemental oxygen when used with compressed oxygen. The **VersaPAP™** device is for patients (ages 5 years and above) who are capable of following directions for Positive Airway Pressure Therapy in a hospital environment. The **VersaPAP™** device is a single patient, multiple use device intended to be used under the supervision of a healthcare professional.

Description

The **VersaPAP™** Positive Airway Pressure (PAP) device is a non-sterile single patient use hand held respiratory therapy device consisting of the Backpiece (A), Mouthpiece (B), Manometer Adapter (C), Manometer Gauge (D), and **EZ Twist** Tubing (E). The **VersaPAP™** Positive Airway Pressure device has the ability to provide supplemental oxygen when used with compressed oxygen.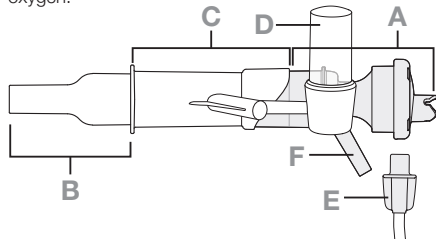

- A – Backpiece
- B – Mouthpiece
- C – Manometer Adapter
- D – Manometer Gauge
- E – **EZ Twist** Tubing
- F – Air Supply Connection

Contraindications

Although no absolute contraindications to the use of PAP therapy have been reported, the following should be carefully evaluated before a decision is made to initiate therapy:

- Patients unable to tolerate the increased work of breathing (acute asthma, COPD)
- Intracranial pressure (ICP) > 20 mm Hg
- Hemodynamic instability
- Recent facial, oral, or skull surgery or trauma
- Acute sinusitis
- Epistaxis
- Esophageal surgery
- Active hemoptysis
- Nausea
- Known or suspected tympanic membrane rupture or other middle ear pathology
- Untreated pneumothorax

WARNING: Use of this device at excessive pressures may have adverse effects. Expert clinical judgment should be exercised in the selection of settings for each individual patient.

Instructions for Use

Before use, ensure these instructions have been read and are kept available at all times.


AIR SUPPLY SET-UP

1. Firmly connect one end of the **EZ Twist** Tubing (E) to a 0-15 L/min flow regulator that is connected to a 50 psi air source (or oxygen if prescribed).
2. Firmly connect the other end of the tubing to the Air Supply Connection (F).

WARNINGS:

- **Do not exceed a flow of 15 l/min or a supply gas inlet pressure of 50 psi; risk of barotrauma.**
- **Do not exceed the clinically prescribed oxygen flow rate when using compressed oxygen as the gas supply, risk of hyperoxia.**
- **Close supervision is required when the product is used with children. It contains parts that may present a choking or strangulation hazard.**

INITIAL SET-UP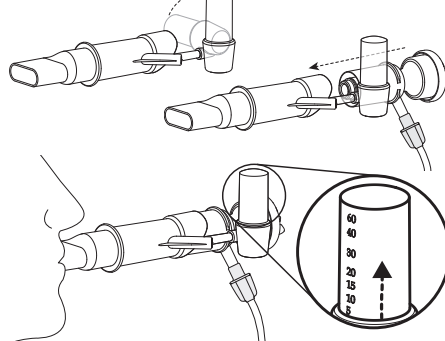

1. Fit the Backpiece to the Manometer Adapter, observing the orientations as shown in the picture.
2. Set the flow rate to 5 L/min on the flow regulator.
3. Instruct the patient to relax while performing diaphragmatic breathing.
4. Place mouthpiece in patient's mouth and instruct them to breathe slowly and easily against the pressure from the device.
5. While monitoring airway pressure with the Manometer Gauge (D), slowly increase the flow rate until desired expiratory airway pressure is reached.

6. Slow inhalations and exhalations by the patient will maintain adequate airway pressure during the breathing cycle.

NOTE: The pressure will be highest during exhalation and lowest during inhalation.

PAP THERAPY PROCEDURE


1. Ensure the flow meter is set as described in previous steps, or to the flow setting at the last therapy session.
2. Check that flow can be felt coming out of the Mouthpiece.
3. Have the patient sit in a comfortable position.
4. Instruct the patient to relax while performing diaphragmatic breathing.
5. Have the patient put the mouthpiece in mouth and close lips around it to ensure an effective seal (a nose clip can be used if needed).
6. Have the patient perform breathing through the device for both inhalation and exhalation.
7. Periodically check for a suitable breathing rate and patient tolerance, adjust the flow rate if needed.
8. Have the patient maintain breathing through the device for the entire therapy session to accomplish clinical goals.

NOTE: A therapy session is intended to last no more than 20 minutes.

Cleaning The Device

Clean the device after each use or as required.

CAUTION: Do not autoclave, device may be damaged.

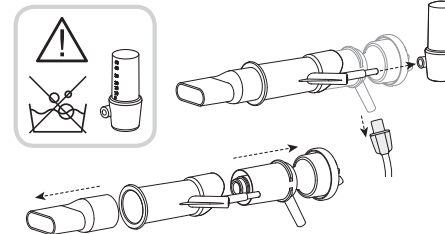1. Separate the **EZ Twist** Tubing from the Air Supply Connection.
2. Remove the Manometer Gauge (D) from the Manometer Adapter (C).
 - **CAUTION:** The Manometer Gauge cannot be cleaned or disinfected. It is a sealed unit.
 - The **EZ Twist** Tubing and Manometer Gauge can be cleaned by wiping the outside surfaces with a tissue moistened with 70% isopropyl alcohol.
3. Rinse the device with warm tap water.
4. Separate the Mouthpiece from the Manometer Adapter, and separate the Manometer Adapter from the Backpiece.

5. The Mouthpiece, Manometer Adapter and Backpiece can be cleaned by using the following method:

- Wash disassembled parts in a warm water/liquid dish detergent solution for 10 minutes, rinse with clean water, and shake to remove any excess.
- Rinse the parts again under running tap water for 1 min.

6. Air dry thoroughly (or dry with a lint free cloth) before reassembly.
7. Ensure Manometer Gauge is reassembled back onto the Manometer Adapter prior to next use.

NOTES

- Replace the **VersaPAP™** device after 7 days of use or immediately if damaged.
- Not made with BPA (Bisphenol A) or natural rubber latex.

Technical Information

Manometer Accuracy	0 – 30 cm H ₂ O = ± 2 cm H ₂ O 30 – 40 cm H ₂ O = ± 3 cm H ₂ O 40 – 60 cm H ₂ O = ± 5 cm H ₂ O
Operating Conditions	+59° to 113°F (15° to 45°C) Ambient to 85% RH
Storage Conditions	-40° to 140°F (-40° to 60°C) Ambient to 75% Relative Humidity (RH)

Reorder Number	68510
----------------	-------

Manufactured for:

Monaghan Medical Corporation
5 Latour Avenue, Suite 1600, Plattsburgh, NY 12901 USA
Tel. 800-833-9653 • Fax 518-561-5088
customerservice@monaghanmed.com
www.monaghanmed.com

Manufactured by:
Trudell Medical International
725 Third Street, London, ON,
Canada N5V 5G4

Quantity: 1

121001-001 Rev B. ™/® are trademarks and registered trademarks of Trudell Medical International. Copyright © Trudell Medical International 2018. All rights reserved. Printed in Canada.

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