


## AEROCHAMBER PLUS® Z STAT® ANTI-STATIC VALVED HOLDING CHAMBER (aVHC)

### Quick Start Guide How to Use Your Chamber


**1** Remove the inhaler cap.

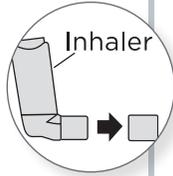


**2** Shake the inhaler.



**3** Insert the inhaler into the back of the chamber.

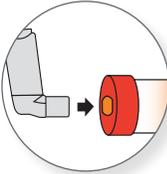


**4** Apply mask to face.

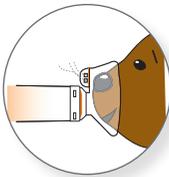


**5** Press inhaler and count for 5 to 6 breaths.



**6** Repeat steps 2 to 5 as prescribed.

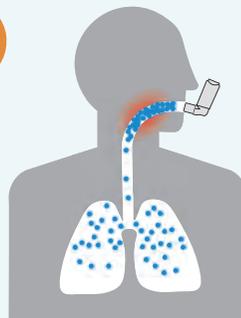

**Using Inhaler Alone:**  
More medication collects in mouth and throat (potential for side effects).

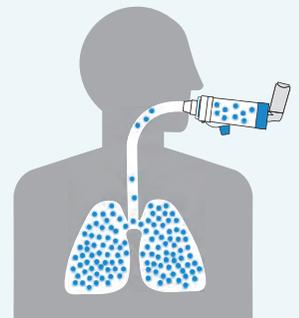

**Using Inhaler with aVHC:**  
Medication reaches lungs.



## AEROCHAMBER PLUS® Z STAT® ANTI-STATIC VALVED HOLDING CHAMBER (aVHC)



Small Mask


Medium Mask

Mouth piece

Large Mask

### Quick Start Guide Cleaning Your Chamber


**1** Remove only the backpiece.

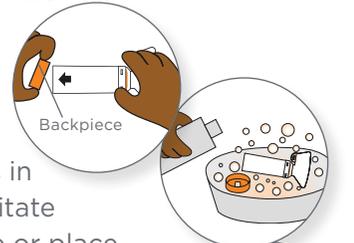


**2** Soak 15 minutes in soapy water, agitate gently and rinse or place in dishwasher on top rack.

Dishwasher instructions can be found with the complete instructions packaged with your AEROCHAMBER PLUS® Z-STAT® aVHC.

**3** Shake out water and dry vertically.

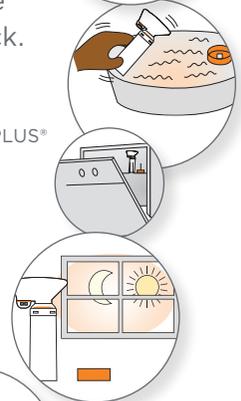

**4** Reassemble the backpiece.

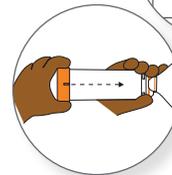

These are abbreviated use and cleaning instructions, please remember to read the complete instructions that are packaged with your AEROCHAMBER PLUS® Z-STAT® anti-static Valved Holding Chamber (aVHC).