


ABBREVIATED SET-UP STEPS

- 1 Remove the mouthpiece from the AEROBIKA® OPEP device and replace with the manometer adapter.
- 2 Insert the backpiece of the VersaPAP® device onto the other port of the AEROBIKA® OPEP device.
- 3 Securely connect the EZ TWIST Tubing to the Air Supply Connection and the flow meter.



ABBREVIATED USE INSTRUCTIONS

- 1 Turn on the flow meter to 5 lpm (flow range is 5-15 lpm).
- 2 Inhale deeply and then exhale for 3-4 times longer than on inhalation. Use the manometer gauge to adjust flow or resistance to maintain a pressure between 15-20 cm H₂O.
- 3 Inhale and exhale for the prescribed amount of time. Perform 2-3 huff coughs for every 10-20 breaths.

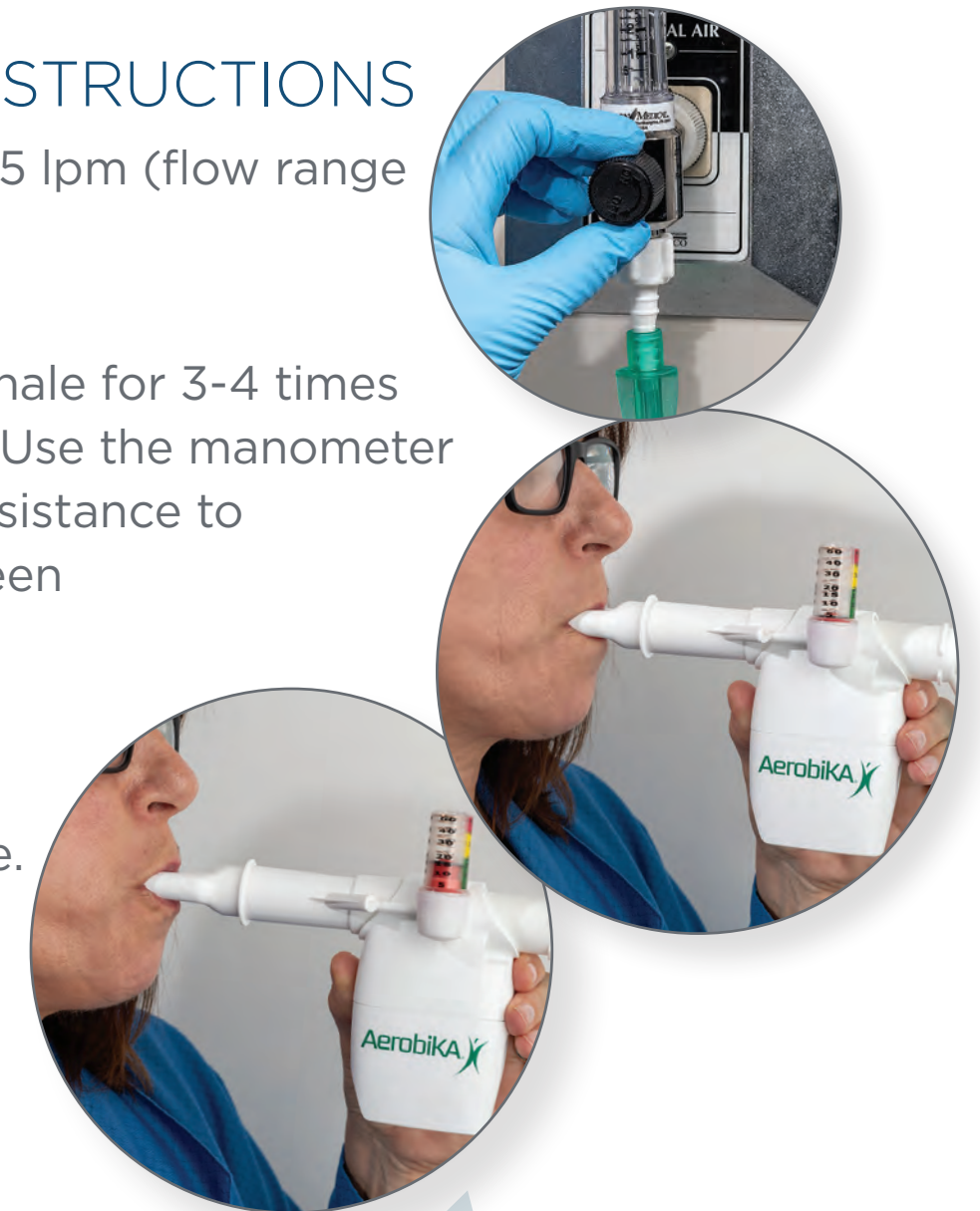

- 4 Repeat combination therapy as directed.

These are abbreviated instructions for use. Please remember to read the complete instructions packaged with your AEROBIKA® OPEP and VersaPAP® devices.

	Therapy Selection	PEP	OPEP	PAP	OPEP+PAP
Ventilation	Recruits collapsed airways (prevent/reduce atelectasis)	●	●	●	●
	Reduces hyperinflation	●	●	●	●
	Augments inspiratory flow to achieve larger tidal volumes			●	●
	Maintains positive pressure on inspiration			●	●
Airway Clearance	Maintains positive pressure on expiration	●	●	●	●
	Gets air behind secretions (mobilization)	●	●	●	●
	Changes properties of secretions		●		●
Aerosol Delivery	Improves aerosol delivery (due to open airways and decreased secretions)	●	●	●	●
	Minimal drug loss when used concurrently with aerosol delivery		●		
Other	Provides oxygen therapy			●	●

Monaghan Medical Corporation 518.561.7330 phone
 Plattsburgh, NY 12901 800.833.9653 toll free
 518.561.5088 fax



monaghanmed.com