**Manufactured for:**  
 Monaghan Medical Corporation  
 Plattsburgh, NY 12901 USA  
 Tel. 800-833-9653 • Fax 518-561-5088  
 customerservice@monaghanmed.com  
 www.monaghanmed.com

**Manufactured by:**  
 Trudell Medical International  
 725 Baransway Drive,  
 London, ON, Canada N5V 5G4

Quantity: 1


## Indications for Use

### VERSAPAP® DEVICE (ALONE)

The **VersaPAP®** device is indicated for the treatment and prevention of atelectasis. The **VersaPAP®** device also has the ability to provide supplemental oxygen when used with compressed oxygen. The **VersaPAP®** device is for patients (ages 5 years and above) who are capable of following directions for Positive Airway Pressure Therapy in a hospital environment. The **VersaPAP®** device is a single patient, multiple use device intended to be used under the supervision of a healthcare professional.

### VERSAPAP® DEVICE WITH AEROBIKA® OPEP DEVICE (COMBINED)

The Combined **VersaPAP®** and **AEROBIKA®** OPEP device is intended for use as a Positive Airway Pressure (PAP) device and a Positive Expiratory Pressure (PEP) device. The combined device has the ability to provide supplemental oxygen when used with compressed oxygen. The combined device is for patients (ages 5 years and above) who are capable of following directions for generating exhalation flow of 10 lpm for 3 – 4 seconds. The combined device is a single patient, multiple use device intended to be used in a hospital environment under the supervision of a healthcare professional.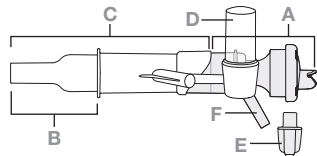

A – Backpiece  
 B – Mouthpiece  
 C – Manometer Adapter  
 D – Manometer Gauge  
 E – Tubing  
 F – Air Supply Connection

## Description

The **VersaPAP®** Positive Airway Pressure (PAP) device is a non-sterile single patient use hand held respiratory therapy device consisting of the Backpiece (A), Mouthpiece (B), Manometer Adapter (C), Manometer Gauge (D), and tubing (E). The **VersaPAP®** Positive Airway Pressure device has the ability to provide supplemental oxygen when used with compressed oxygen.

## Contraindications

Although no absolute contraindications to the use of PAP therapy have been reported, the following should be carefully evaluated before a decision is made to initiate therapy:

- Patients unable to tolerate the increased work of breathing (acute asthma, COPD)
- Intracranial pressure (ICP) > 20 mm Hg
- Hemodynamic instability
- Recent facial, oral, or skull surgery or trauma
- Acute sinusitis
- Epistaxis
- Esophageal surgery
- Active hemoptysis
- Nausea
- Known or suspected tympanic membrane rupture or other middle ear pathology
- Untreated pneumothorax

**WARNING: Use of this device at excessive pressures may have adverse effects. Expert clinical judgment should be exercised in the selection of settings for each individual patient.**

## Instructions for Use

Before use, ensure these instructions have been read and are kept available at all times.


### AIR SUPPLY SET-UP

1. Firmly connect one end of the tubing (E) to a 0-15 L/min flow regulator that is connected to a 50 psi air source (or oxygen if prescribed).
2. Firmly connect the other end of the tubing to the Air Supply Connection (F).

### WARNINGS:

- Do not exceed a flow of 15 l/min or a supply gas inlet pressure of 50 psi; risk of barotrauma.
- Do not exceed the clinically prescribed oxygen flow rate when using compressed oxygen as the gas supply, risk of hyperoxia.
- Close supervision is required when the product is used with children. It contains parts that may present a choking or strangulation hazard.

### INITIAL SET-UP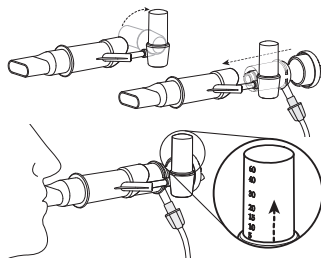

1. Fit the Backpiece to the Manometer Adapter, observing the orientations as shown in the picture.
2. Set the flow rate to 5 L/min on the flow regulator.
3. Instruct the patient to relax while performing diaphragmatic breathing.
4. Place mouthpiece in patient's mouth and instruct them to breathe slowly and easily against the pressure from the device.
5. While monitoring airway pressure with the Manometer Gauge (D), slowly increase the flow rate until desired expiratory airway pressure is reached.
6. Slow inhalations and exhalations by the patient will maintain adequate airway pressure during the breathing cycle.


**NOTE:** The pressure will be highest during exhalation and lowest during inhalation.

## PAP THERAPY PROCEDURE

1. Ensure the flow meter is set as described in previous steps, or to the flow setting at the last therapy session.
2. Check that flow can be felt coming out of the Mouthpiece.
3. Have the patient sit in a comfortable position.
4. Instruct the patient to relax while performing diaphragmatic breathing.
5. Have the patient put the mouthpiece in mouth and close lips around it to ensure an effective seal (a nose clip can be used if needed).
6. Have the patient perform breathing through the device for both inhalation and exhalation.
7. Periodically check for a suitable breathing rate and patient tolerance, adjust the flow rate if needed.
8. Have the patient maintain breathing through the device for the entire therapy session to accomplish clinical goals.

## PAP + OPEP TANDEM THERAPY PROCEDURE

The **VersaPAP®** device may be used with the **AEROBIKA®** OPEP device for Tandem Therapy. Please review the instructions for use provided with that device prior to using these instructions.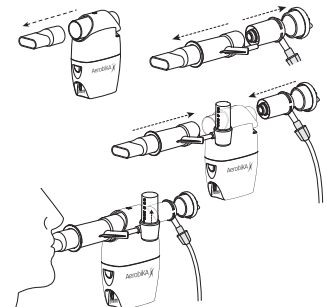

1. Remove the mouthpiece from the **AEROBIKA®** OPEP device and set aside in a safe location.
2. Separate the Manometer Adapter (C) from the Backpiece (A) by gently pulling them apart. Leave the mouthpiece of the PAP device in place.
3. Insert the Manometer Adapter onto the **AEROBIKA®** OPEP device. It will fit only one of the OPEP ports.
4. Insert the Backpiece onto the other port of the **AEROBIKA®** OPEP device.

5. Follow the **VersaPAP®** device set-up instructions set out in 'PAP THERAPY PROCEDURE' section above.
6. On exhalation, use the proper technique for the **AEROBIKA®** OPEP device, as directed in the Instructions for Use.
7. Have the patient maintain breathing through the combined system for the entire therapy session to accomplish clinical goals.


### NOTES

- A therapy session is intended to last no more than 20 minutes twice each day.
- When used as a combined PAP + OPEP device, it is not intended for use with a nebulizer.
- Exhalation resistance may be higher when the **VersaPAP®** device and **AEROBIKA®** OPEP device are used in tandem. Use the Manometer Gauge to determine if flow or resistance level need to be adjusted.
- Tandem Therapy can be used for up to 7 days. The **AEROBIKA®** OPEP device can be used longer – see package instructions.

## Cleaning the VersaPAP® Device

Clean the device after each use or as required.

**CAUTION:** Do not autoclave, device may be damaged.

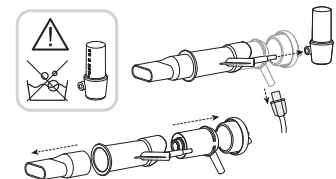1. Separate the tubing from the Air Supply Connection.
2. Remove the Manometer Gauge (D) from the Manometer Adapter (C).

**CAUTION:** The Manometer Gauge cannot be cleaned or disinfected. It is a sealed unit.

- The tubing and Manometer Gauge can be cleaned by wiping the outside surfaces with a tissue moistened with 70% isopropyl alcohol.
3. Rinse the device with warm tap water.
  4. Separate the Mouthpiece from the Manometer Adapter, and separate the Manometer Adapter from the Backpiece.
  5. The Mouthpiece, Manometer Adapter and Backpiece

can be cleaned by using the following method:

- Wash disassembled parts in a warm water/liquid dish detergent solution for 10 minutes, rinse with clean water, and shake to remove any excess.
  - Rinse the parts again under running tap water for 1 min.
6. Air dry thoroughly (or dry with a lint free cloth) before reassembly.
  7. Ensure Manometer Gauge is reassembled back onto the Manometer Adapter prior to next use.

### NOTES

- Replace the **VersaPAP®** device after 7 days of use or immediately if damaged.
- Not made with BPA (Bisphenol A) or natural rubber latex.

## Technical Information

Manometer Accuracy	0 – 30 cm H <sub>2</sub> O = ± 2 cm H <sub>2</sub> O 30 – 40 cm H <sub>2</sub> O = ± 3 cm H <sub>2</sub> O 40 – 60 cm H <sub>2</sub> O = ± 5 cm H <sub>2</sub> O
Operating Conditions	+59° to 113°F (15° to 45°C) Ambient to 85% RH
Storage Conditions	-40° to 140°F (-40° to 60°C) Ambient to 75% Relative Humidity (RH)

**AeroVerse®**  
 SYSTEM OF PRODUCTS

## REORDER NUMBERS

Part Number	Description	Quantity
68510	<b>VersaPAP®</b> Positive Airway Pressure Device with <b>EZ TWIST</b> Tubing	10/cs
68500	<b>VersaPAP®</b> Positive Airway Pressure Device with <b>EZ TWIST</b> Tubing	100/cs
68310	<b>VersaPAP®</b> Positive Airway Pressure Device with Universal Tubing	10/cs
68300	<b>VersaPAP®</b> Positive Airway Pressure Device with Universal Tubing	100/cs

121011-001 Rev C. <sup>TM</sup>/<sub>®</sub> trademarks and registered trademarks of Trudell Medical International. Copyright © Trudell Medical International 2019–2021. All rights reserved. Printed in Canada.

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