


Quick Start Guide
How to Use Your Chamber


1 Remove the inhaler cap.

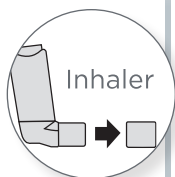


2 Shake the inhaler.



3 Insert the inhaler into the back of the chamber.

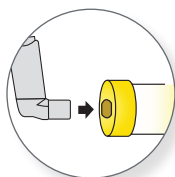


4 Apply mask to face.

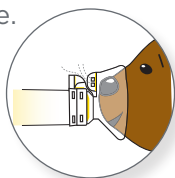


AEROCHAMBER PLUS®
FLOW-VU® aVHC

People who use a valved holding chamber with their inhaler have **better control**.¹

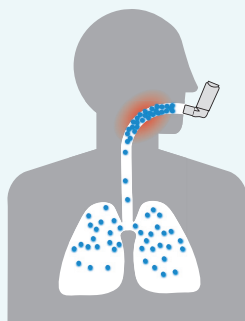


Inhaler alone with poor technique

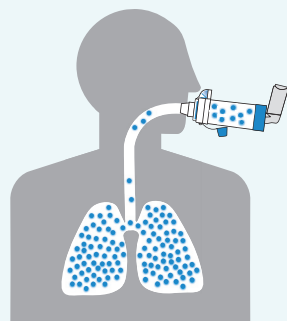

Inhaler with valved holding chamber

5 Press inhaler and count for 5 to 6 breaths.

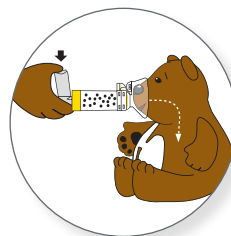

6 Repeat steps 2 to 5 as prescribed.




Small Mask

Medium Mask

Mouth piece

Intermediate Mask

Large Mask

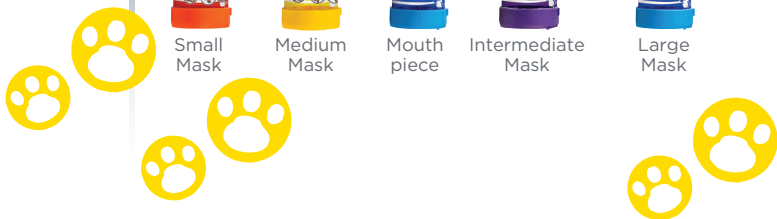




AEROCHAMBER PLUS[®] FLOW-VU[®] aVHC




Quick Start Guide Cleaning Your Chamber

1 Remove front and back pieces.

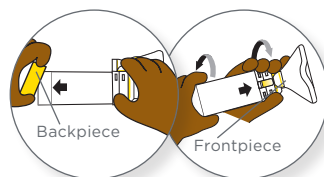


2 Soak 15 minutes in soapy water, agitate gently and rinse or place in dishwasher on top rack.

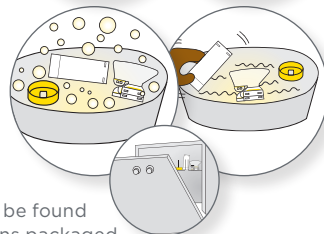


Dishwasher instructions can be found with the complete instructions packaged with your AEROCHAMBER PLUS[®] FLOW-VU[®] aVHC.

3 Shake out water and dry vertically.

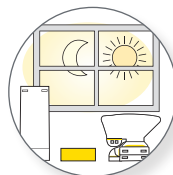

4 Reassemble the chamber.

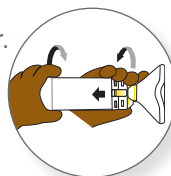

These are abbreviated use and cleaning instructions, please remember to read the complete instructions that are packaged with your AEROCHAMBER PLUS[®] FLOW-VU[®] Anti-Static Valved Holding Chamber (aVHC).