FILTERED MOUTHPIECE KIT

for the AEROECLIPSE® BAN™ Nebulizer

INTENDED USE

The filter kit is intended for single patient use as an accessory to a small volume nebulizer. The device can be used in a hospital, home or clinic with patients 5 years and above to minimize the amount of medical aerosol exhaled into the air.

BEFORE EACH USE

Carefully examine the filter kit, removing any foreign objects and replace immediately if it is damaged or has missing parts.

- 1 Replace the mouthpiece of the AEROECLIPSE® BAN™ Nebulizer with the assembled filter kit.


NOTE: Set aside the AEROECLIPSE® BAN™ Nebulizer mouthpiece. It can be reinstalled after the filter kit has been removed.

- 2 Instruct the patient to tightly seal their lips around the mouthpiece, and inhale and exhale normally through the device. Verify movement of the green feedback button (A).

NOTE: The patient should only use the mouthpiece for both inhalation and exhalation and may consider using a nose clip to prevent potential exhaled infectious materials to the environment.

- 3 Ensure the filter kit's exhalation flow path (B) is not blocked.
- 4 Continue treatment using the instructions supplied with the AEROECLIPSE® BAN™ Nebulizer.

TO REPLACE THE FILTER

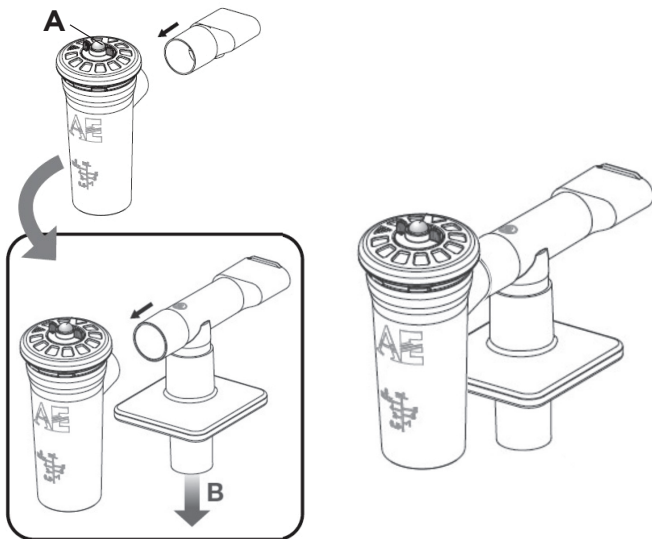- 1 Remove the filter from the mouthpiece tee.
- 2 Firmly attach a new filter to the bottom of the mouthpiece tee.

CLEANING INSTRUCTIONS

- 1 Separate the mouthpiece tee from the filter.
- 2 The mouthpiece tee may be cleaned in soap and water allowing parts to soak for 15 minutes. Agitate gently. Rinse parts in clean water.
- 3 Shake to remove excess water and allow to air dry thoroughly before attaching a filter.

HIGH-LEVEL DISINFECTION INSTRUCTIONS

Prior to disinfection, follow cleaning instructions. Do not dry or reassemble the device.

CIDEX⁺ OPA HIGH LEVEL DISINFECTANT: Soak the tee in an active solution of Cidex⁺ OPA as per the manufacturer's instructions. This method was validated using an active solution of Cidex⁺ OPA. Rinse well and allow to air dry thoroughly before attaching a filter.

CAUTION

- The filter is single use only. Replace after each use.
- The filter cannot be cleaned or disinfected.

NOTES

- Single Patient Use. Do not share.
- Mouthpiece tee can be used for up to 7 days.
- This product was not made with natural rubber latex.
- Storage temperature: 32°-77°F (0°-25°C)

Manufactured by:
Monaghan Medical Corporation
153 Industrial Boulevard, Plattsburgh, NY 12901
www.monaghanmed.com

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