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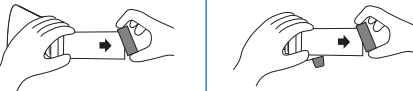
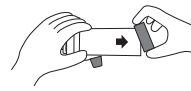
Manufactured by / Fabricado por:

Monaghan Medical Corporation.
Plattsburgh, NY 12901
www.monaghanmed.com





INDICATIONS FOR USE

This product is intended to be used by patients who are under the care or treatment of a physician or licensed healthcare professional. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers. The intended environments for use include the home, hospitals and clinics.

BEFORE EACH USE


This product can be used right out of package. Before use, ensure these instructions and the instructions supplied with the Metered Dose Inhaler (MDI) have been read.

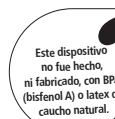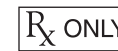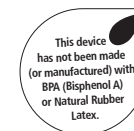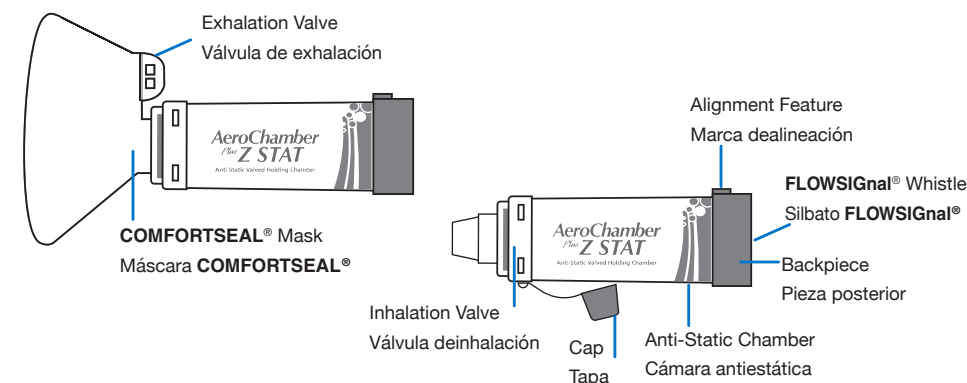FIGURE 1 / FIGURA 1

© 2022 Monaghan Medical Corporation™ and ® are trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. This device was Made in the USA from US and Canadian parts.

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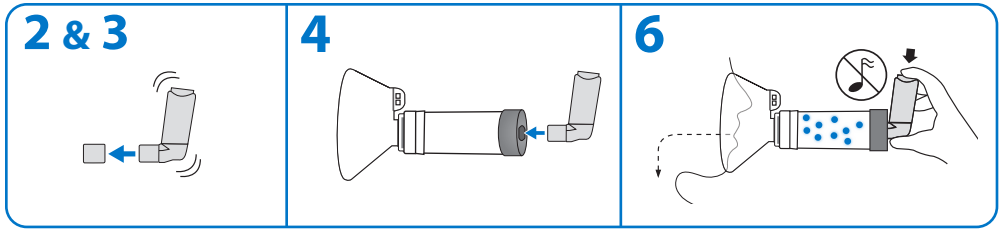
INSTRUCTIONS FOR USE CHAMBER WITH MASK

- Carefully examine the product for damage, missing parts or foreign objects. Remove any foreign objects prior to use. The product should be replaced IMMEDIATELY if there are any damaged or missing parts. If necessary, use the Metered Dose Inhaler (MDI) alone until a replacement is obtained. If the patient's symptoms worsen, please seek immediate medical attention.
- Remove cap(s) from the MDI
- Shake the MDI immediately before each use as per the instructions supplied with the MDI.
- Insert the MDI into the backpiece of the chamber.
- Apply mask to face and ensure an effective seal.
- Breathe out gently and depress the MDI at the beginning of a slow inhalation. Maintain seal for 5–6 breaths after the MDI is depressed. Slow down inhalation if you hear the FLOWSIGNAL® Whistle sound. **Administer one (1) puff at a time.**
- Follow instructions supplied with the MDI on how long to wait before repeating steps 3 – 6 as prescribed.

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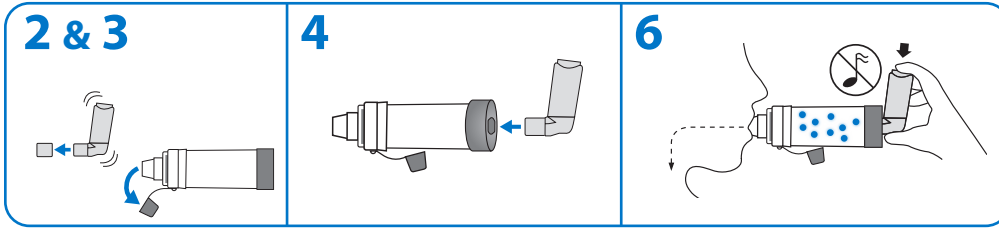INSTRUCTIONS FOR USE MOUTHPIECE CHAMBER

- Carefully examine the product for damage, missing parts or foreign objects. Remove any foreign objects prior to use. The product should be replaced IMMEDIATELY if there are any damaged or missing parts. If necessary, use the Metered Dose Inhaler (MDI) alone until a replacement is obtained. If the patient's symptoms worsen, please seek immediate medical attention.
- Remove cap(s) from the MDI and chamber.
- Shake the MDI immediately before each use as per the instructions supplied with the MDI.
- Insert the MDI into the backpiece of the chamber.
- Put mouthpiece into mouth and close lips around it to ensure an effective seal.
- Breathe out gently and depress the MDI at the beginning of a slow inhalation. Breathe in slowly and deeply through the mouth until a full breath has been taken. Hold breath for 5–10 seconds, if possible. Otherwise, keep lips tight on the mouthpiece to maintain seal for 2–3 breaths after the MDI is depressed. Slow down inhalation if you hear the FLOWSIGNAL® Whistle sound. **Administer one (1) puff at a time.**
- Follow instructions supplied with the MDI on how long to wait before repeating steps 3–6 as prescribed.

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CLEANING INSTRUCTIONS FOR MASK AND MOUTHPIECE CHAMBERS

This product can be used right out of the package and then cleaned weekly.

CHAMBER WITH MASK	CHAMBER WITH MOUTHPIECE	
1 		Remove the backpiece only. Do not tamper with valve during cleaning or disassemble the product beyond what is recommended or damage may result. For mask product, do not remove mask.
2 		Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently. Rinse parts in clean water. ; E Dishwasher safe under the following conditions: Avoid heavy or other cycles that include heated dry over 158°F; Parts must be placed on the top rack ONLY; Secure product face up as pictured; Use dishwasher detergent and rinse aid; Do not boil or sterilize; see CAUTION #1.
3 		Shake out excess water from the parts and allow to air dry in a vertical position. Ensure parts are dry before reassembly.
4 		To reassemble, press firmly to attach the backpiece. For mouthpiece models, the protective cap should always be placed on the mouthpiece when the product is not in use.

Notes

- SINGLE PATIENT USE.
- Storage and operating range 5°C – 40°C (41°F – 104°F) at 15 to 95% relative humidity.
- Product may need to be replaced after 12 months of use. Environmental conditions, storage and proper cleaning can affect product life span.
- Do not share this medical device.
- Clarity of the chamber is a result of the properties of the STATBAN® anti-static material.
- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Dishwashing with overly dirty dishes is not recommended.
- Dishwasher validated up to 158°F (70°C).

Cautions

- PRODUCT MAY BE PERMANENTLY DAMAGED IF BOILED, STERILIZED OR CLEANED IN A DISHWASHER AT TEMPERATURE ABOVE 158°F (70°C).
- Do not leave the chamber unattended with children.
- Federal (USA) law restricts the sale of this device on or by the order of a physician.