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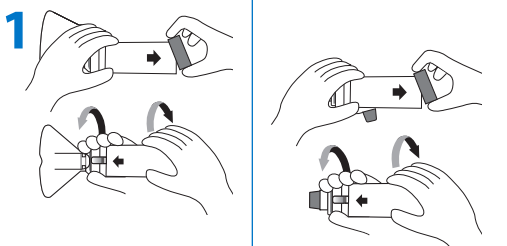
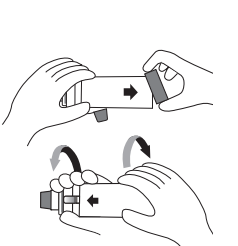
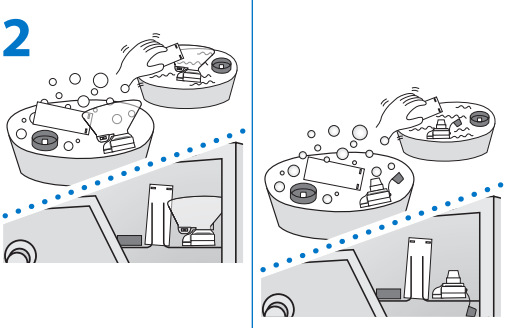
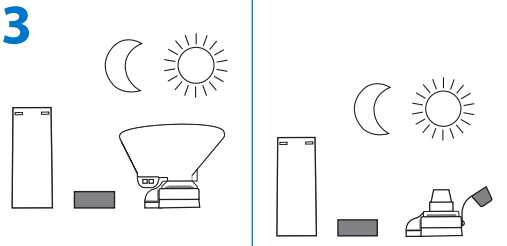
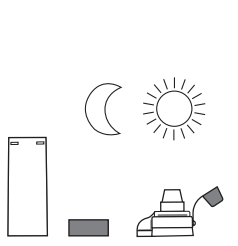
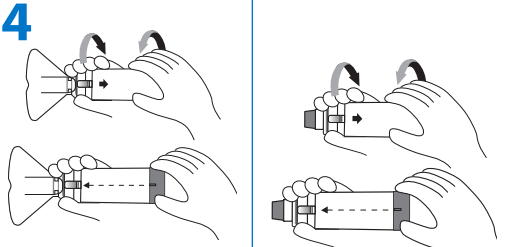
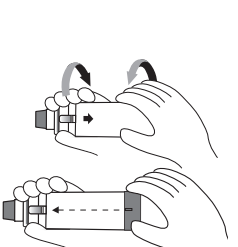
Notes

- SINGLE PATIENT USE
- The device is provided non-sterile and should not be boiled or sterilized.
- Storage and operating range 41°F–104°F (5°C–40°C) at 15 to 95% relative humidity.
- Product should be replaced after 12 months of use.
- Do not share this medical device.
- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Dishwashing with overly dirty dishes is not recommended.
- Dishwasher validated up to 158°F (70°C).
- If cleaning in a dishwasher use a rinse aid.

⚠ Cautions

- **Do not leave the chamber unattended with children. This is not a toy.**
- **Product may be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 158°F (70°C).**
- **Federal (USA) law restricts the sale of this device on or by the order of a physician.**

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Manufactured by / Fabricado por:

monaghan

Monaghan Medical Corporation.
5 Latour Ave., Suite 1600, Plattsburgh, NY 12901
www.monaghanmed.com

INDICATIONS FOR USE

This product is intended to be used by adult and pediatric patients who are under the care or treatment of a physician or licensed healthcare professional. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers and Soft Mist Inhalers. The intended environments for use include the home, hospitals and clinics. It is a single patient, multiple use device.

BEFORE EACH USE

This chamber can be used directly out-of-package. Ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times. Carefully examine the chamber and replace immediately if it is damaged or has missing parts.

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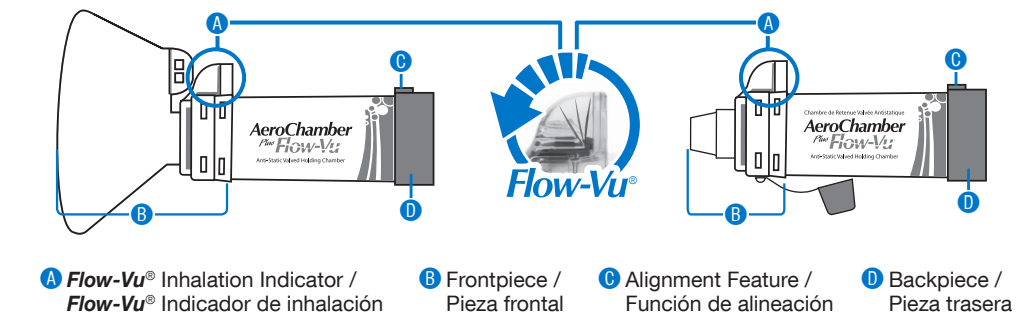
FIGURE / FIGURA 1

This device has not been made (or manufactured) with BPA (Bisphenol A) or Natural Rubber Latex.

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