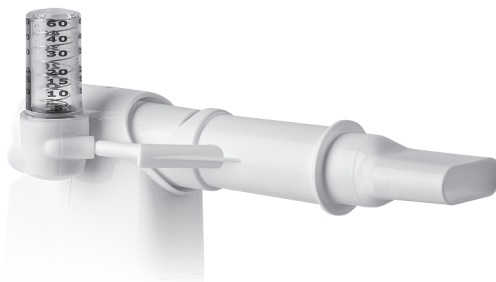
Manometer Accessory

for use with the **Aerobika**® Oscillating Positive Expiratory Pressure Device

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Manufactured by / Fabricado por:
 Trudell Medical International
 725 Baransway Drive, London, Ontario,
 Canada N5V 5G4

Rx Only

Qty: 1
 Cant: 1

110251-001 Rev A. 2024-09-27. *Registered trade marks of Trudell Medical International.
 † trade marks and registered trade marks of their respective companies.
 Copyright © Trudell Medical International 2024. All rights reserved.

EN

INTENDED USE

The manometer is intended to be used as a single patient use accessory to the **Aerobika**® Oscillating Positive Expiratory Pressure (OPEP) device as an indicator for patient expiratory pressure.

INSTRUCTIONS FOR USE

Please read the **Aerobika**® OPEP device Instructions for Use prior to using these instructions.
 An electronic version of these Instructions for Use is available for download at www.monaghanmed.com

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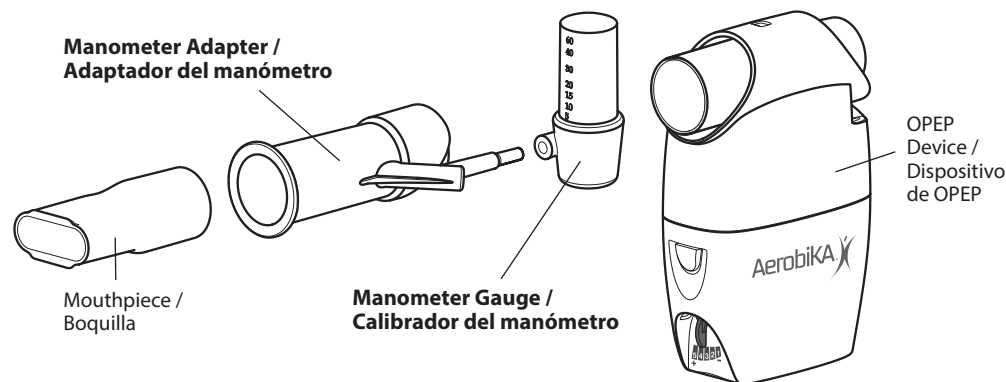
EN

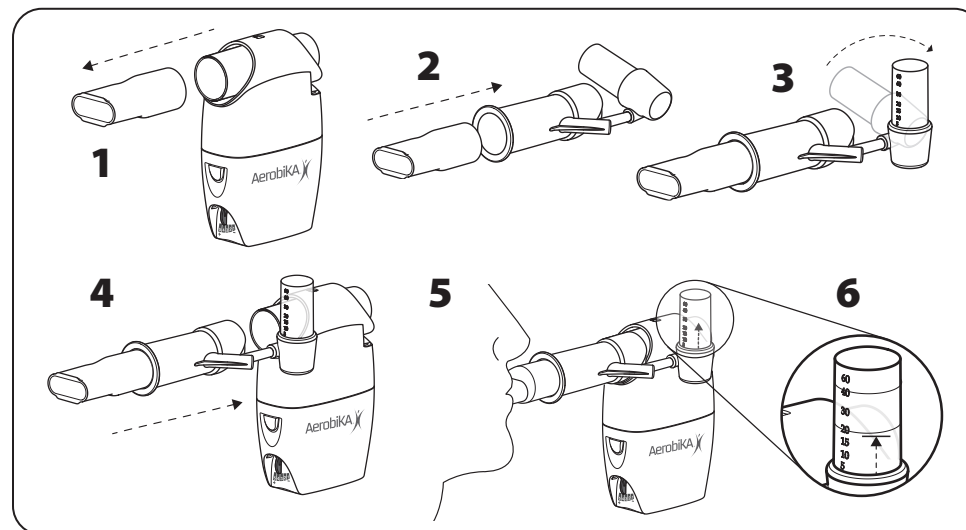
SETTING UP YOUR MANOMETER WITH THE AEROBIKA® OPEP DEVICE

Before use, ensure these instructions have been read and are kept available at all times. If you have questions about the performance or usability of this product, please contact your healthcare professional prior to use.

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1 Remove the mouthpiece from the **Aerobika**® OPEP device.

2 Insert the mouthpiece onto the end of the manometer adapter.

3 Once the mouthpiece is inserted, ensure that the manometer gauge is securely attached in place and rotate so the numbers are facing upward.

4 Insert the adapter onto the **Aerobika**® OPEP device in place of the mouthpiece as shown.

Note: Follow the proper technique for using your **Aerobika**® OPEP device. Refer to the Instructions for Use provided with the device.

5 As you exhale actively, the manometer gauge indicator will rise based on your exhalation pressure.

6 For each exhalation, maintain the pressure level recommended by your healthcare professional.

⚠ CAUTION: If you feel dizzy or lightheaded, or have any other discomfort during treatment, discontinue therapy and contact your physician or healthcare provider.

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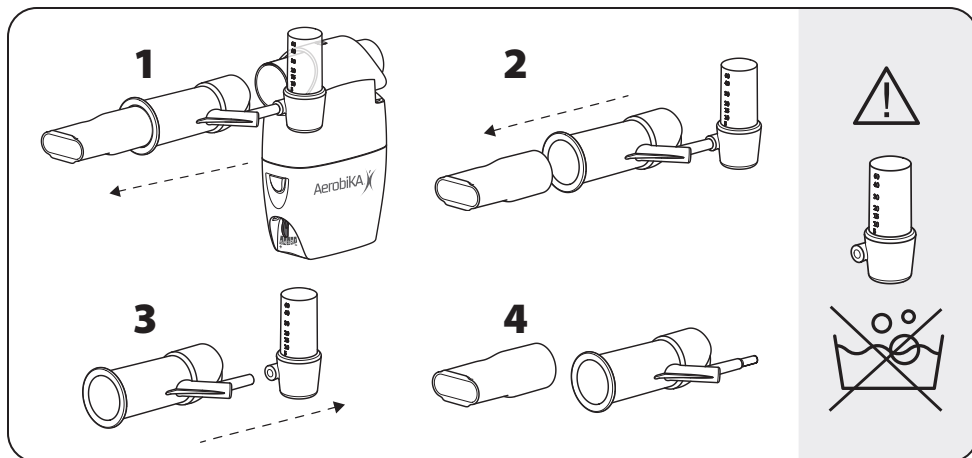
EN

CLEANING YOUR MANOMETER ADAPTER

The manometer adapter may be cleaned daily.

1 Remove the manometer adapter from the **Aerobika®** OPEP device.

2 Remove the mouthpiece from the manometer adapter.

3 Remove the manometer gauge from the manometer adapter and wipe down the exterior of the manometer gauge using a wipe (i.e., Accel PREvention, SoluPrep, Loris, Clorox Healthcare).

NOTE: The manometer gauge can be cleaned a maximum of 183 times (once daily for 6 months). At the end of 6 months, the sticker on the manometer gauge may begin to peel, this will not impact performance of the device.

4 The mouthpiece and manometer adapter may be cleaned daily in soap and water, allowing parts to soak for 15 minutes. Agitate gently. Rinse parts in clean water.

OR

Place the mouthpiece and manometer adapter in a dishwasher basket on the top rack. Wash in a normal cycle. Dishwashing with overly dirty dishes is not recommended.

Shake to remove excess water and allow to air dry thoroughly before reassembly.

⚠ CAUTION: The manometer gauge cannot be cleaned by soaking. It is a sealed unit.

DISINFECTING YOUR MANOMETER ADAPTER

The mouthpiece and manometer adapter may be disinfected weekly using any of the following methods:

A Boil for 5 minutes.

B Microwave Steam Bag (according to manufacturer's instructions).

C Soak in 70% isopropyl alcohol for 5 minutes. Rinse well.

D Soak in 3% hydrogen peroxide for 30 minutes. Rinse well.

Allow to air dry thoroughly before reassembly.

⚠ CAUTION: The manometer gauge cannot be disinfected with these methods.

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EN

⚠ CAUTIONS

- Do not leave the manometer unattended with children. It is not a toy.
- Do not soak, rinse, or sterilize the manometer gauge as these procedures may leave harmful residues or cause malfunction or inaccuracy of the device. The design and material used are not compatible with conventional cleaning and sterilization procedures.
- The manometer gauge color coding scale is only for visual support. The correct patient expiratory pressure must be determined by a medical professional.
- Make sure the manometer gauge is firmly attached to the adapter. Ensure the adapter is firmly attached to the **Aerobika®** OPEP device to avoid leaking.
- The manometer should not be used with any other device or on its own.
- Federal (USA) law restricts the sale of this device on or by the order of a physician.

ⓘ NOTES

SINGLE PATIENT USE.

- Product should be replaced after 6 months of use, upon expiry, or immediately if damaged.
- This product has not been made (or manufactured) with BPA (Bisphenol A) or natural rubber latex.
- Do not share this medical device.
- Store the manometer with your **Aerobika®** OPEP device in a dry, dust-free place avoiding continuous exposure to sunlight.

RECYCLING

The manometer gauge is not recyclable, please remove the manometer gauge prior to recycling.

DISPOSAL

The manometer (adapter and gauge) can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in your State.

TECHNICAL INFORMATION

Flow Values for Color Coded Scale Ranges	0 – 20 cm H ₂ O is the 'green' zone 21 – 40 cm H ₂ O is the 'yellow' zone 41 – 60 cm H ₂ O is the 'red' zone
Storage Conditions	-40° to 140°F (-40° to 60°C), ambient to 75% Relative Humidity (RH)
Operating Conditions	+59° to 113°F (+15° to 45°C), ambient to 85% RH
Measurement Error	0 – 30 cm H ₂ O = ± 2 cm H ₂ O 30 – 40 cm H ₂ O = ± 3 cm H ₂ O 40 – 60 cm H ₂ O = ± 5 cm H ₂ O

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RECICLAJE

The manometer gauge is not recyclable. Please remove the manometer gauge prior to recycling.

ELIMINACIÓN

The manometer (adapter and gauge) can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in your State.

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