Instructions for Use

Before use, wash your hands then check the nebulizer. Replace immediately if any damage is noticed. Ensure these instructions and those supplied with the medication have been read.

1 Open the nebulizer by rotating the top counter clockwise. Pour prescribed medication into the cup. Maximum fill volume is 6 mL. Close the nebulizer, making sure the green nozzle cover is inside, then rotate the top until it stops.

2 Place the mouthpiece (2a) or corrugated tube with mouthpiece (2b) onto the nebulizer. If a mask is required, the nebulizer should be used with the masks listed in reorder number section. Refer to the instructions packaged with the mask.

3 Firmly attach one end of the tubing to the nebulizer. Take care not to tip the device or medication could spill. Attach the other end of the tubing to an air supply set within the following range:

Range	Flow	Pressure
Min	4 [L/min]	13 [psi]
Max (recommended)	8 [L/min]	30 [psi]

When the air supply is on, ensure the device is creating a mist.

4 Place the mouthpiece in your mouth. Ensure your lips are sealed around the edge. If using a mask, position the mask lightly against the face. Inhale the aerosol breathing normally. Exhale normally through the mouthpiece (or mask). Treatment time may be longer with facemask use. Continue treatment until sputter.

Description	Part Number	Case Size
MC 300 [®] Nebulizer – EZ Twist Tubing	67150	50/case
MC 300 [®] Nebulizer – Universal Tubing	67750	50/case
MC 300 [®] Nebulizer – with Corrugated tubing and mouthpiece, Universal Tubing	67450	50/case
Disposable Aerosol Mask – Small	65750	50/case
Disposable Aerosol Mask – Medium	65950	50/case
Disposable Aerosol Mask – Large	65850	50/case

Manufactured for:
 Monaghan Medical Corporation
 Plattsburgh, NY 12901 USA
 Tel. 800-833-9653 • Fax 518-561-5088
 customerservice@monaghanmed.com
 www.monaghanmed.com

Manufactured by:
 Trudell Medical International
 725 Baransway Drive
 London, ON, Canada N5V 5G4

Cleaning Instructions

For home, hospital or clinic

- The exterior surfaces of the nebulizer may be wiped with a gauze lightly moistened with either distilled water or 70% isopropyl alcohol (solvent)
- Remove the tubing from the bottom of the device, and disassemble the nebulizer into its parts (mouthpiece, corrugated tube (if supplied), green nozzle cover, top and cup). Corrugated tubing should be fully extended for cleaning and until dry.
- Cleaning of the nebulizer is recommended after each aerosol treatment

Any of the following four methods may be used:

A Sterile Water: Rinse the disassembled nebulizer parts thoroughly in sterile water. Shake to remove excess water.

B 70% Isopropyl Alcohol: Wipe the outside of the nebulizer with a tissue dipped in 70% isopropyl alcohol. Submerge the four parts in 70% isopropyl alcohol for 1 minute. Shake to remove any excess isopropyl alcohol.

C Air Dry: Disassemble the nebulizer and shake out any excess drug.

D Hand Washing: Wash the disassembled parts in a warm water and liquid dish soap solution. Rinse in clean warm water.

- After cleaning, inspect the device to ensure it is clean. If any residual soil is detected, repeat the selected cleaning step.
- For all methods, always allow to air dry completely (or dry with a lint free cloth) before reassembling. It is recommended the nebulizer bottom be briefly reconnected to the air supply to clear the air pathway.

NOTE:

- Regular cleaning will help prevent build-up of residual medication, which can affect aerosol generation
- Always follow accepted hospital procedures for cleaning between uses with the same patient

Tubing

Occasionally you may notice condensation on the inside of your tubing. This is normal. To remove the condensation, connect one end of the tubing to your air supply and allow air to run through the tube without the nebulizer attached. This will force air through the tubing, drying the inside. To clean the exterior, wipe

with a lightly moistened gauze with solvent (such as distilled water or 70% isopropyl alcohol). Ensure tubing is dry prior to storage or use.

Intended Use

The nebulizer is intended to be used with pediatric (ages 2 years and above) and adult patients, who are under the care of a licensed healthcare provider or physician. The device is designed to aerosolize prescribed medication for inhalation by a patient in the hospital, clinic or home care environment. The nebulizer is a single patient use device.

The corrugated tube is an accessory to the MC 300[®] Nebulizer. It is designed to work in conjunction with the MC 300[®] Nebulizer to extend the patient interface away from the nebulizer. The accessory is a single patient use device that is intended to be used with either mouthpiece or mask for pediatric (ages 2 years and above) and adult patients, who are under the care of a licensed healthcare provider or physician.

Duration of Use

The nebulizer is a single patient, disposable device for use up to 7 days. The device is provided non-sterile and should not be reprocessed, autoclaved or boiled.





The corrugated tube with mouthpiece is a single patient, disposable accessory for use up to 7 days. The accessory is provided non-sterile and should not be reprocessed, autoclaved, or boiled.

WARNING

- This is a single patient use device.
- Close supervision is required when the product is used with children. It contains parts that may present a choking or strangulation hazard.

CAUTIONS

- Federal law restricts this device to sale by or on the order of a physician.
- For patients under the age of 5, a mask shall be used.
- Not made with BPA (Bisphenol A) or Natural Rubber Latex.
- Failure to properly clean the device may affect the performance.

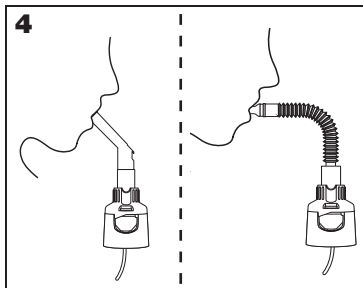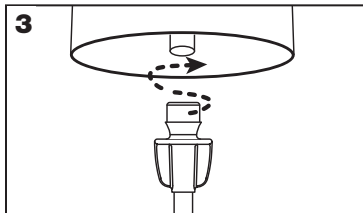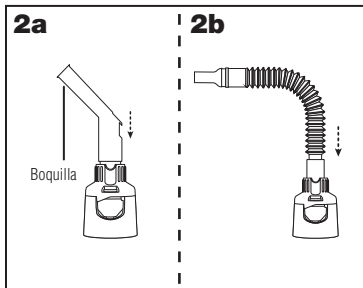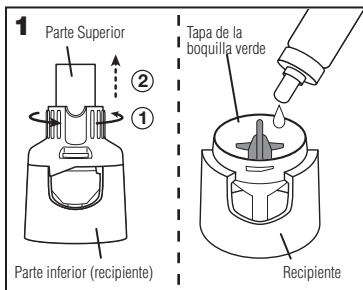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Fabricado por:
Trudell Medical International
725 Baransway Drive, London, ON,
Canada N5V 5G4

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