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Aero2go[™]
Chamber

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2 in 1
Chamber +
Protective Case

Aero2go[™]
Chamber

Anti-Static Valved Holding Chamber

Manufactured by:
Trudell Medical International

Manufactured for:

monaghan

Monaghan Medical Corporation
153 Industrial Boulevard,
Plattsburgh, NY 12901
T 800-833-9653
customerservice@monaghanmed.com
www.monaghanmed.com

● INTENDED USE

This product is intended to be used by patients (5+ years) who are under the care or treatment of a physician or licensed healthcare professional. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers. It is a single patient, multiple use device intended for home use.

● HOW IT WORKS

Using the chamber with your metered dose inhaler will capture and hold the medication until you are ready to breathe it in. This gives you more time to inhale and helps the medication get into your lungs where it is needed to help you.

Warnings

- Carefully look into the chamber. If there is any visible dust or other debris inside, then clean the chamber.
- Replace immediately if the chamber is damaged or has missing parts.

Cautions

- Do not use with Soft Mist Inhalers (i.e. Respimat[®]), Dry Powder Inhalers or Nebulizers. The chamber has not been designed to deliver these medications so you may not get the proper dose.
- Failure to follow instructions may affect medication delivery or cause inhalation issues.

● USING YOUR CHAMBER

Shake

inhaler immediately before use (remove cap)

Insert

inhaler into back of the chamber

Press

inhaler as you start to breathe in **slowly and deeply**

Notes

- Ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times.
- Take 1 puff at a time.
- The chamber should be replaced after 12 months of use.
- **SINGLE PATIENT USE.**
- Federal (USA) law restricts the sale of this device on or by the order of a physician.
- This product has not been made (or manufactured) with BPA (Bisphenol A), natural rubber latex, phthalates, lead, or PVC.

Tips

- To help improve delivery of medication to your lungs, after a deep breath, try to hold your breath for 2-5 seconds before exhaling.
- If you are not able to take a deep breath or hold it in, you can breathe through the chamber 2 -3 times to inhale your medication.

● 2 IN 1 CHAMBER + PROTECTIVE CASE FOR YOUR INHALER

Your inhaler can be stored inside the chamber.


- Remove the cap from the inhaler when storing it inside the chamber.
- Chamber is enclosed on both ends helping to prevent debris from entering.
- Designed to fit most commonly prescribed inhalers. (Note: inhaler is sold separately)

Caution

- Not designed for use with round mouthpiece inhalers such as Proventil[®] and Alvesco[®]. Use may cause premature wear or damage to device.

To open chamber: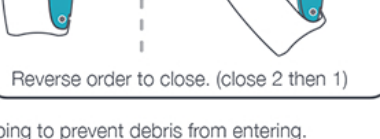

Reverse order to close. (close 2 then 1)

Warning

- Ensure you (and if applicable your child) can open the chamber before storing your inhaler inside.
- The chamber is not intended for use by children less than 5 years of age.

● CLEANING YOUR CHAMBER

The chamber can be used directly out of the package and should be cleaned weekly.

1 Lift the cap and open the chamber to remove the inhaler.

2 Wash the chamber in warm, clean water with liquid dish detergent, agitating gently. Rinse in clean water.

OR Place on top rack of dishwasher as shown. Run the dishwasher on a normal or light cycle. Do not heat dry.

3 Shake out excess water and allow to air dry. If you require the chamber immediately, dry gently with a clean dish towel.

Notes

- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a clean soft cloth.
- Do not boil or sterilize.
- Dishwashing with overly dirty dishes is not recommended.
- If cleaning in a dishwasher, use a rinse aid.
- For cleaning the inhaler, follow the instructions supplied with the inhaler.

Caution

- Product may be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 70°C.

An electronic version of this Product Insert is available for download at www.monaghanmed.com

Qty = 1

R_x ONLY

REPLACE CADA 12 MESES

123071-001 Rev 02, 2024-08-29. TM/_® trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. © 2024 Monaghan Medical Corporation. This device is made with US and Canadian parts.