

IMPROVE  
INHALER MEDICATION  
DELIVERY WITH



**Aero2go™**  
**Chamber**


Anti-Static Valved Holding Chamber

**INHALERS RELEASE AEROSOL VERY QUICKLY,  
WHICH CAN CAUSE PROBLEMS LIKE:**


- Uncontrolled or untreated symptoms from less medicine settling in the lungs
- A sore throat, raspy voice or 'thrush' (a mouth yeast infection) from medication hitting the back of the throat
- Wasted medication from your inhaler

**CHAMBERS HOLD MEDICATION UNTIL YOU'RE  
READY TO BREATHE IT IN. USE YOUR CHAMBER  
EVERY TIME YOU USE YOUR INHALERS!**

Significantly  
more  
medication  
deposits in  
the back of  
the throat.

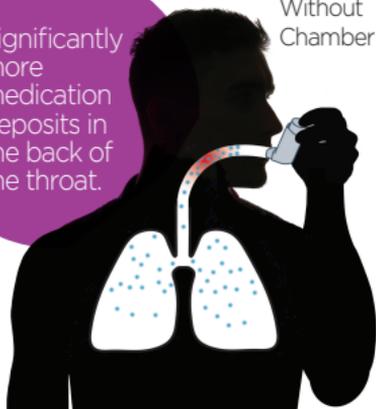

Optimize  
medication  
delivery to  
the lungs!

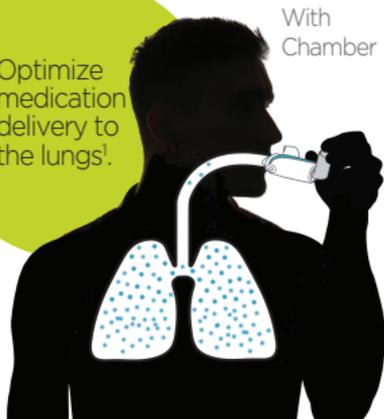

Part of the AEROCHAMBER<sup>®</sup> Family of Valved Holding Chambers

## Easy To Use



**Shake**  
the inhaler



**Insert** the inhaler  
into the back of  
the chamber



**Press** the inhaler  
as you breathe  
in slowly and deeply

## 2 in 1 Chamber + Protective Case

Your inhaler can be stored inside the chamber.

To open chamber:



**Reverse** order to close.  
(close **2** then **1**)

**Note:** If the inhaler doesn't fit, just flip it over and fit inside the chamber. Inhaler prescribed separately.

## Easy to clean + all one piece

**Chambers** should  
be cleaned weekly.

Hand Wash



Dishwasher Safe



OR

References: 1. Gillen M *et al.* Pulm Pharmacol Ther 2018;52:7-17.

Words or phrases accompanied by <sup>™</sup> and <sup>®</sup> are trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. © 2025 Monaghan Medical Corporation. 70003-00, 08/2025

 monaghan

Monaghan Medical  
Corporation  
153 Industrial Blvd.  
Plattsburgh, NY 12901

518.561.7330 **phone**  
800.833.9653 **toll free**  
518.561.5088 **fax**  
**monaghanmed.com**