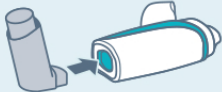
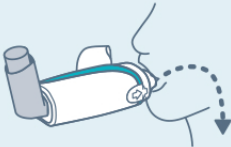


Aero2goTM Chamber

Anti-Static Valved Holding Chamber

HELP IMPROVE INHALER MEDICATION DELIVERY

Easy to Use

- **1 Remove** the inhaler cap.
- **2 Shake** the inhaler.
- **3 Insert** the inhaler into the back of the chamber.
- **4 Seal lips** around the mouthpiece.
- **5 Press** the inhaler, breathe in slowly and hold your breath for 5 to 10 seconds or breathe 2 to 3 times.
- **6 Repeat** steps 2 to 5 as prescribed.

WITHOUT CHAMBER

Significantly more medication deposits in the back of the throat.

WITH CHAMBER

Optimize medication delivery to the lungs¹

INHALERS RELEASE AEROSOL VERY QUICKLY, WHICH CAN CAUSE PROBLEMS LIKE:

- Uncontrolled or untreated symptoms from less medicine settling in the lungs
- A sore throat, raspy voice or 'thrush' (a mouth yeast infection) from medication hitting the back of the throat
- Wasted medication from your inhaler

Chambers hold medication until you're ready to breathe it in.
Use your chamber every time you use your inhalers!

2 in 1 Chamber + Protective Case

Your inhaler can be stored inside the chamber.

To open chamber:



Reverse order to close.
(close **2** then **1**)

Note: If the inhaler doesn't fit, just flip it over and fit inside the chamber. Inhaler prescribed separately.

Easy to clean + all one piece



Hand Wash

OR



Dishwasher Safe

Chambers should be cleaned weekly.

