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Aerobika®

Oscillating Positive
Expiratory Pressure Device
with Manometer Accessory

Manufactured for:

Monaghan Medical Corp.
153 Industrial Boulevard
Plattsburgh, NY 12901
☎ +1-800-833-9653
📠 +1-518-561-5088
customerservice@monaghanmed.com
www.monaghanmed.com

Manufactured by:

Trudell Medical International
725 Baransway Drive
London, Ontario, Canada N5V 5G4

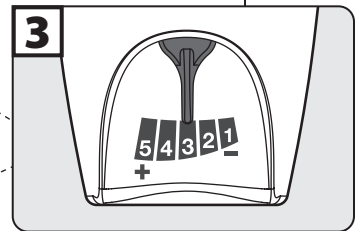
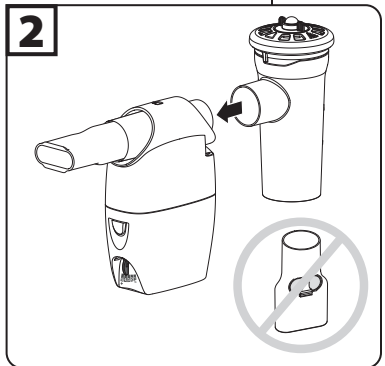
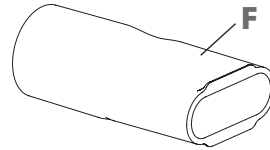
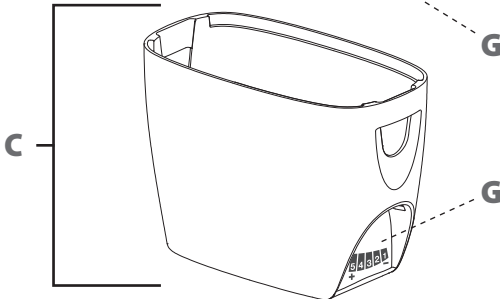
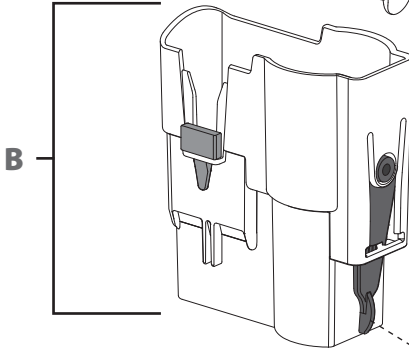
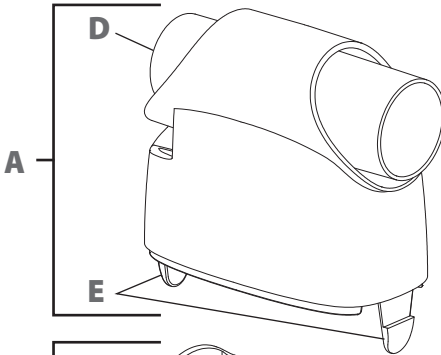
1

EN

- A Top Case
- B Valve Cartridge
- C Bottom Case
- D Nebulizer Port
- E Attachment Tabs
- F Mouthpiece
- G Resistance Indicator
- H Instructions for use

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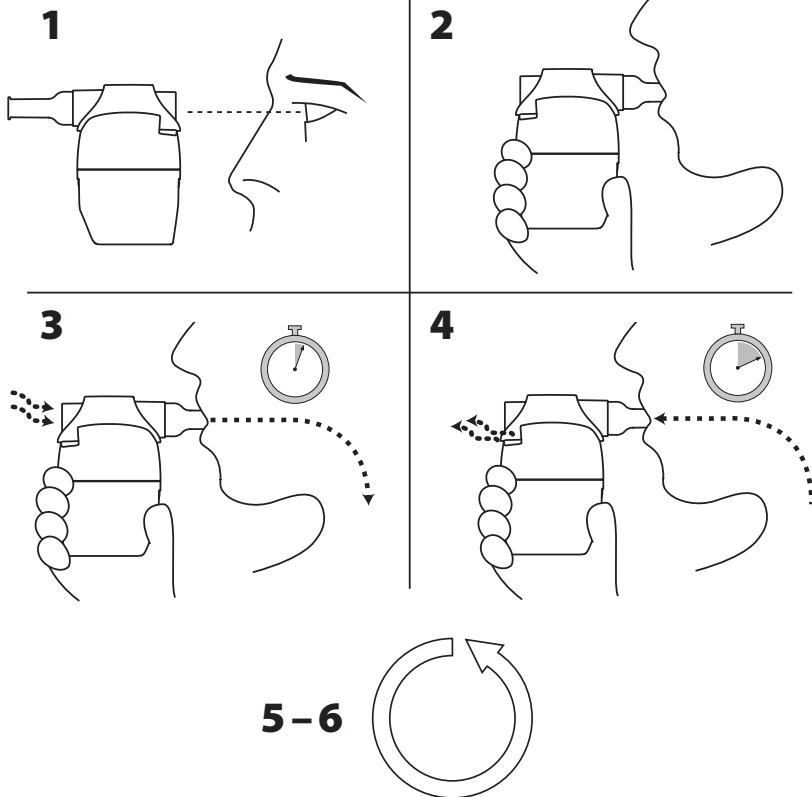
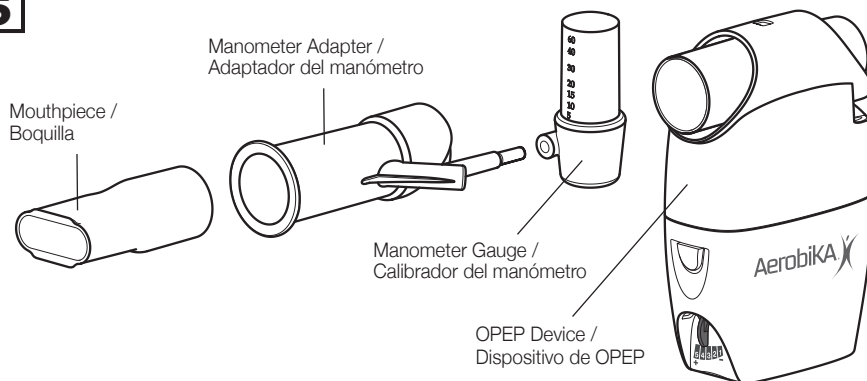
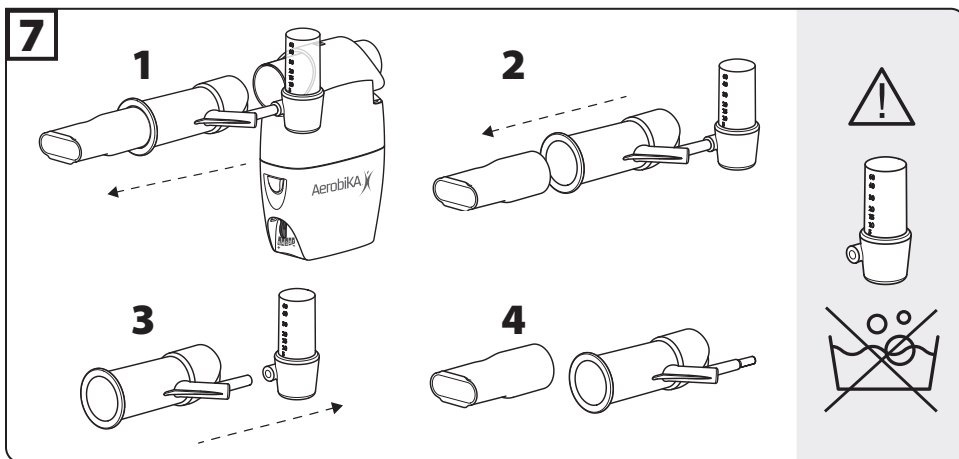
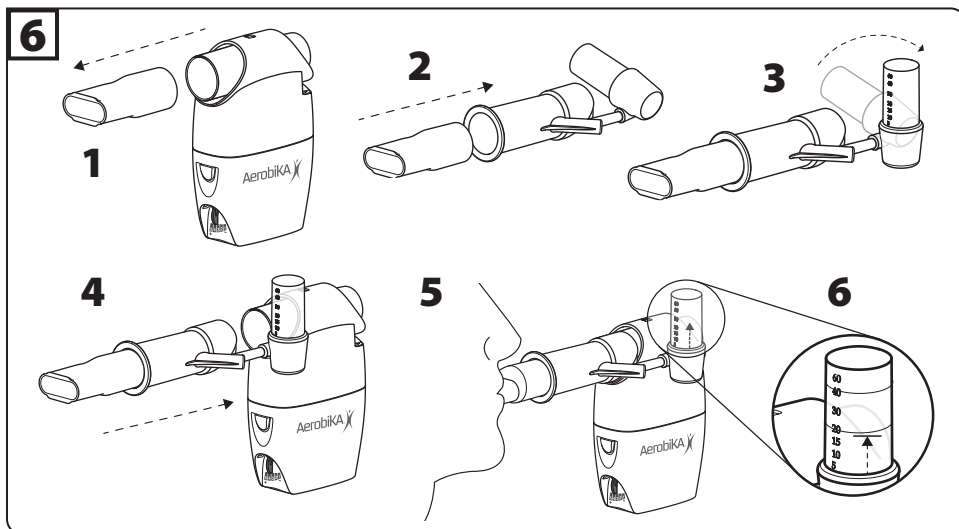
Aerobika® Device

INTENDED USE

The **Aerobika®** Oscillating Positive Expiratory Pressure Therapy System is intended for use as a Positive Expiratory Pressure (PEP) device. The **Aerobika®** Oscillating PEP device may also be used simultaneously with nebulized aerosol drug delivery. The device is intended to be used by patients capable of generating exhalation flow of 10 lpm for 3–4 seconds.

⚠ CAUTIONS

- Do not leave the device unattended with children. It is not a toy.
- Do not press attachment tabs when the case is disassembled. Doing so could break the tabs and prevent proper assembly of the device.
- Ensure contraindications are observed.
- Federal (USA) law restricts the sale of this device on or by the order of a physician.

CONTRAINDICATIONS

Use of the **Aerobika®** Oscillating PEP device is not recommended for the following conditions

- Patients unable to tolerate the increased work of breathing
- Intracranial pressure (ICP) > 20 mm Hg
- Hemodynamic instability (blood pressure instability)
- Recent facial, oral, or skull surgery or trauma
- Acute sinusitis
- Epistaxis (bleeding nose)
- Esophageal surgery
- Nausea
- Active hemoptysis (bleeding from lungs)
- Untreated pneumothorax (untreated collapsed lung)
- Known or suspected tympanic membrane rupture or other middle ear pathology

⚠ WARNING

Use of this device at excessive pressures may have adverse effects. Expiratory pressures above 20 cm H₂O in patients sensitive to increased transpulmonary pressure may develop one or more of the adverse side effects listed below. Expert clinical judgment should be exercised in the selection of the appropriate setting for each individual patient. Failure to match the appropriate resistance setting on the Resistance Indicator with the patient's expiratory flow may result in failure to achieve therapeutic objectives of oscillatory PEP therapy or one or more adverse side effects listed below.

Adverse reactions may include

- Increased work of breathing that may lead to hypoventilation and hypercarbia
- Increased cranial pressure
- Cardiovascular compromise
 - Myocardial ischemia
 - Decreased venous return
- Air swallowing with increased likelihood of vomiting and aspiration
- Claustrophobia
- Pulmonary barotraumas

SETTING UP THE AEROBIKA® OSCILLATING PEP

For device descriptions see Figure [1].

Before use, ensure these instructions have been read and are kept available at all times. If you have questions about the performance or usability of this product, please contact your healthcare professional prior to use.

DETERMINING PROPER RESISTANCE SETTING

The **Aerobika®** Oscillating PEP device should be set up by a healthcare professional using the following steps:

1 Start with the Resistance Indicator at the middle setting. (Figure [3])

2 With the patient in a comfortable seated position, have them inhale through the device taking a deeper breath than normal, but not to total lung capacity, then exhale briskly but not forcefully through the device.

3 Resistance should be set so exhalation lasts for 3–4 times longer than inhalation. Resistance can be increased by moving the indicator to the left. To reduce resistance, move the indicator to the right.

Note: Selection of the proper resistance setting will produce the desired inspiratory:expiratory flow ratio of 1:3 or 1:4 for 10–20 minutes without excess fatigue.

4 Patients should be reevaluated periodically or with any change in status. As airways become less congested, more resistance will be needed to maintain inspiratory:expiratory ratios.

⚠ CAUTION: The resistance setting should be adjusted as recommended by your healthcare professional.

PROPER TECHNIQUE FOR USING YOUR AEROBIKA® OSCILLATING PEP (FIGURE [4])

Treatments should be conducted twice daily or as directed by your healthcare professional. The **Aerobika®** Oscillating PEP can be used in any orientation. It is recommended that treatments be done in a comfortably seated position.

1 Before use, carefully examine the device.

- If there is any visible dust or other debris inside, then clean the device. Replace immediately if it is damaged or has missing parts.
- Always ensure the Resistance Indicator is set as prescribed by your healthcare professional.
- Gently press the two halves together to make sure the Attachment Tabs are engaged.

2 Put the mouthpiece in mouth and close lips around it to ensure an effective seal. Make sure your hand does not block the exhalation path on the back of the device.

3 Inhale through the device taking a deeper breath than normal, but do not totally fill your lungs. Hold your breath for 2–3 seconds before exhaling.

4 Exhale actively, but not forcefully, through the device. Ideally, exhalation should last 3–4 times longer than inhalation. Keep your cheeks flat and firm to maximize treatment effectiveness. Ensure you maintain a good seal on the mouthpiece for the duration of your treatment.

5 Continue taking deep breaths and long exhalations for 10–20 breaths, or as instructed by your healthcare professional. After at least 10 breaths, perform 2–3 'huff coughs' to clear your airways. Your healthcare professional will instruct you on the proper technique. For maximum efficacy, try to suppress coughing until the end of your treatment.

6 Repeat steps 3–5 for 10–20 minutes or as prescribed by your healthcare professional. The entire procedure should be performed at least twice a day on a regular

basis, and increased to 3–4 times per day if you are producing a lot of mucus.

The goal is to keep your lungs clear and ventilatory capacity up. Continue to use your treatments on a regular basis even if you are bringing up just a little mucus.

⚠ CAUTION: If you feel dizzy or lightheaded, or have any other discomfort during treatment, discontinue therapy and contact your physician or healthcare provider.

HOW TO USE YOUR AEROBIKA® OSCILLATING PEP WITH YOUR NEBULIZER

The **Aerobika®** Oscillating PEP device may also be used with a small volume nebulizer with a 22 mm adapter. Your healthcare professional will advise which medication to use for combined treatments.

Medications which open your airways (eg. albuterol) or help to thin mucus (eg. hypertonic saline) would be good choices to use with your **Aerobika®** Oscillating PEP because they would help remove or thin the mucus in your lungs. Medications you want to stay in your lungs, like antibiotics, should be taken after you have completed your treatment.

1 Ensure you have read and understood the instructions provided with your nebulizer prior to starting your treatment.

2 Ensure your nebulizer is set-up correctly and medication has been added to the nebulizer cup. Remove the mouthpiece (or mask) from the nebulizer.

3 Attach the nebulizer to the Nebulizer Port of the **Aerobika®** Oscillating PEP, as shown in Figure 2.

4 Switch on the compressed air source, and complete your combined treatment using the breathing technique outlined above.

ⓘ NOTE:

- No adapter is required if used with the **AEROECLIPSE®** brand of Breath Actuated Nebulizer.
- Ensure the nebulizer remains upright for proper nebulization.
- Always ensure the **Aerobika®** Oscillating PEP mouthpiece is used otherwise the device will not work properly. (See Figure 2)
- If used in combination with a nebulizer, the device should be cleaned before next use to remove any residue that may have deposited on exhalation.

CLEANING AND DISINFECTING YOUR AEROBIKA® OSCILLATING PEP DEVICE

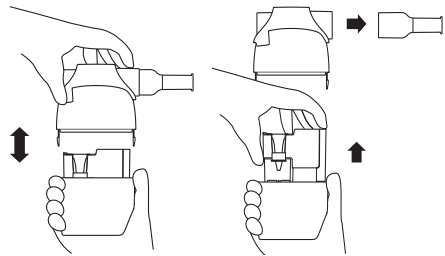
Healthcare providers should always use clinical judgment to select the appropriate Cleaning (1) and Disinfection (2) method and frequency, based on the severity of the patient's disease state.

1 CLEANING INSTRUCTIONS

The **Aerobika®** Oscillating PEP device will disassemble into four parts for cleaning: the mouthpiece, top case, bottom case and valve cartridge. If necessary, the

Aerobika® Oscillating PEP may be cleaned after each treatment.

A TAKING THE DEVICE APART



- Press the Attachment Tabs on either side of the device. Holding the tabs in, gently pull up on the top case until the two parts separate.
- Remove the valve cartridge. **Do not disassemble any part of the cartridge.**
- To remove the mouthpiece, grip firmly and rotate while pulling away from the device.

B CLEANING THE DEVICE


SOAP AND WATER

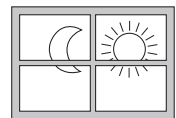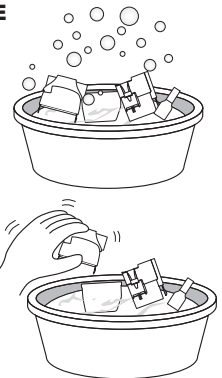- Wash the four parts in a solution of lukewarm clean water and liquid dish detergent, allowing the parts to soak for 15 minutes. Agitate gently.
- Rinse in warm water.
- To dry, shake the parts to remove any excess water. Allow parts to air dry thoroughly before you put the parts back together.

OR DISHWASHER

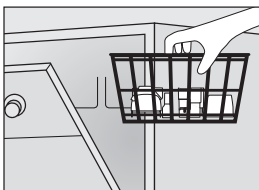- The four disassembled parts may be washed in a dishwasher by securing parts in a basket on the top rack. Wash in a normal wash cycle with dishwasher detergent ensuring use of rinse aid (eg. Jet Dry®).
- If excess water remains, shake the parts and allow to air dry thoroughly before reassembly.
- Dishwashing with overly dirty dishes is not recommended.

C REASSEMBLING THE DEVICE

- Insert the valve cartridge into the bottom case. If it does not slip easily into place, rotate 180° and try again.
- Connect the top and bottom cases until you hear both Attachment Tabs “click” (this may occur simultaneously). Ensure the tabs are engaged by firmly pressing the two halves together.
- Reattach the mouthpiece.

2 DISINFECTION INSTRUCTIONS

The **Aerobika®** Oscillating PEP device may be disinfected weekly using any of the following methods. Prior to disinfection, follow cleaning instructions. Do not dry or reassemble the device.

HIGH LEVEL DISINFECTION

If considered clinically necessary for the patient's disease state, the device may be cleaned **after each use** and disinfected **once each treatment day**. **STERILE water must be used for rinsing.**

A BOILING

- Place the four disassembled parts in a pot when boiling point has been reached and allow to boil for 5 to 15 minutes.

Note: Place a small rack, on the bottom of the pot to prevent plastic parts from coming in contact with the hot bottom.

- Remove from water and allow parts to cool and air dry thoroughly before reassembly.


Note: Tap water can be high in minerals which may result in scale build-up in the device. Depending on the amount of time you boil the device, a more frequent vinegar soak may be required to reduce scale build-up in the device. Please see the chart below for the suggested vinegar soak frequency to keep your device performing well.

Boil Duration	Vinegar Soak Frequency
5 minutes	Monthly
5+ minutes	Every 2 weeks

Disassemble the device and soak in a solution of 1 cup vinegar and 1 cup water for 1 hour, then wash as normal.

B MICROWAVE STEAM BAG

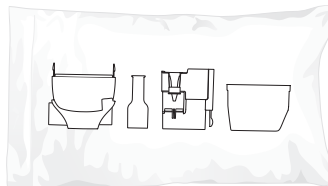The four disassembled parts may be disinfected in the microwave using a steam cleaning bag (i.e. Quick-Clean® MicroSteam® bag). Follow the directions for use closely for each manufacturer of these cleaning bags, ensuring you follow the manufacturer's instructions. Position parts in the bag as shown:

Remove parts from the bag and allow to cool and air dry thoroughly before reassembly.

Important: Do not use the steam bag method of cleaning if the microwave has known “hot spots”. Follow the manufacturer's steam bag instructions carefully. Pay close attention to the amount of water to add and your microwave's wattage. Failure to follow the instructions may result in permanent damage to the device.

C ISOPROPYL ALCOHOL (70%)

The four disassembled parts may be soaked in 70% isopropyl alcohol for 5 minutes. Rinse well and allow to air dry thoroughly before reassembly.

D HYDROGEN PEROXIDE (3%)

The four disassembled parts may be soaked in 3% hydrogen peroxide for 30 minutes. Rinse well and allow to air dry thoroughly before reassembly.

E BLEACH (Not recommended for high level disinfection)

The four disassembled parts may be soaked in 1:50 bleach to water solution (approximately 1 tablespoon bleach in 3¼ cups water) for 3 minutes. Rinse well and allow to air dry thoroughly before reassembling. You may substitute a respiratory equipment disinfectant (such as Milton®, Dodie®, Control III® or Cidex®) for the bleach solution by following the manufacturer's instructions for use.

⚠ CAUTION: Failure to follow the cleaning or disinfection instructions may have an adverse effect on device performance.

RECYCLING

The **Aerobika®** OPEP device is recyclable. Recycling may be limited in your jurisdiction or local regulations may not permit it. Please consult with your community's recycling guidelines to determine availability of curbside recycling or

recycling at a drop-off facility. Clean your device prior to recycling.

DISPOSAL

The **Aerobika**® Oscillating PEP device can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in your state.

NOTES

- SINGLE PATIENT USE.
- Product should be replaced after 12 months of use or immediately if damaged.
- This product has not been made (or manufactured) with BPA (Bisphenol A), natural rubber latex, phthalates, lead, or PVC.
- Do not share this medical device.
- If you are unable to exhale through the device, check the resistance setting or try cleaning the device thoroughly.
- Store the **Aerobika**® Oscillating PEP device in a dry, dust-free place avoiding continuous exposure to sunlight.
- Use temperature range: +15°C to +40°C (59°F to 104°F)
- Storage temperature range: -40°C to +60°C (104°F to 140°F)

Manometer Accessory

INTENDED USE

The manometer is intended to be used as a single patient use accessory to the **Aerobika**® Oscillating Positive Expiratory Pressure (OPEP) device as an indicator for patient expiratory pressure.

INSTRUCTIONS FOR USE

Please read the **Aerobika**® OPEP device Instructions for Use prior to using these instructions.

An electronic version of these Instructions for Use is available for download at www.monaghanmed.com

SETTING UP YOUR MANOMETER WITH THE AEROBIKA® OPEP DEVICE (FIGURE 6)

Before use, ensure these instructions have been read and are kept available at all times. If you have questions about the performance or usability of this product, please contact your healthcare professional prior to use.

- 1 Remove the mouthpiece from the **Aerobika**® OPEP device.
- 2 Insert the mouthpiece onto the end of the manometer adapter.
- 3 Once the mouthpiece is inserted, ensure that the manometer gauge is securely attached in place and rotate so the numbers are facing upward.
- 4 Insert the adapter onto the **Aerobika**® OPEP device in place of the mouthpiece as shown.

Note: Follow the proper technique for using your **Aerobika**® OPEP device. Refer to the Instructions for Use provided with the device.

5 As you exhale actively, the manometer gauge indicator will rise based on your exhalation pressure.

6 For each exhalation, maintain the pressure level recommended by your healthcare professional.

CAUTION: If you feel dizzy or lightheaded, or have any other discomfort during treatment, discontinue therapy and contact your physician or healthcare provider.

CLEANING YOUR MANOMETER ADAPTER (FIGURE 7)

The manometer adapter may be cleaned daily.

1 Remove the manometer adapter from the **Aerobika**® OPEP device.

2 Remove the mouthpiece from the manometer adapter.

3 Remove the manometer gauge from the manometer adapter and wipe down the exterior of the manometer gauge using a wipe (i.e., Accel PREvention, SoluPrep, Loris, Clorox Healthcare).

Note: The manometer gauge can be cleaned a maximum of 183 times (once daily for 6 months). At the end of 6 months, the sticker on the manometer gauge may begin to peel, this will not impact performance of the device.

4 The mouthpiece and manometer adapter may be cleaned daily in soap and water, allowing parts to soak for 15 minutes. Agitate gently. Rinse parts in clean water.

OR

Place the mouthpiece and manometer adapter in a dishwasher basket on the top rack. Wash in a normal cycle. Dishwashing with overly dirty dishes is not recommended.

Shake to remove excess water and allow to air dry thoroughly before reassembly.

CAUTION: The manometer gauge cannot be cleaned by soaking. It is a sealed unit.

DISINFECTING YOUR MANOMETER ADAPTER

The mouthpiece and manometer adapter may be disinfected weekly using any of the following methods:

- 1 Boil for 5 minutes.
- 2 Microwave Steam Bag (according to manufacturer's instructions).
- 3 Soak in 70% isopropyl alcohol for 5 minutes. Rinse well.
- 4 Soak in 3% hydrogen peroxide for 30 minutes. Rinse well.

Allow to air dry thoroughly before reassembly.

CAUTION: The manometer gauge cannot be disinfected with these methods.

CAUTIONS

- Do not leave the manometer unattended with children. It is not a toy.

- Do not soak, rinse, or sterilize the manometer gauge as these procedures may leave harmful residues or cause malfunction or inaccuracy of the device. The design and material used are not compatible with conventional cleaning and sterilization procedures.
- The manometer gauge color coding scale is only for visual support. The correct patient expiratory pressure must be determined by a medical professional.
- Make sure the manometer gauge is firmly attached to the adapter. Ensure the adapter is firmly attached to the **Aerobika**® OPEP device to avoid leaking.
- The manometer should not be used with any other device or on its own.
- Federal (USA) law restricts the sale of this device on or by the order of a physician.

NOTES

- SINGLE PATIENT USE.
- Product should be replaced after 6 months of use, upon expiry, or immediately if damaged.
- This product has not been made (or manufactured) with BPA (Bisphenol A) or natural rubber latex.
- Do not share this medical device.
- Store the manometer with your **Aerobika**® OPEP device in a dry, dust-free place avoiding continuous exposure to sunlight.

RECYCLING

The manometer gauge is not recyclable, please remove the manometer gauge prior to recycling.

DISPOSAL

The manometer (adapter and gauge) can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in your State.

TECHNICAL INFORMATION

Flow Values for Color Coded Scale Ranges	0 – 20 cm H ₂ O is the 'green' zone 21 – 40 cm H ₂ O is the 'yellow' zone 41 – 60 cm H ₂ O is the 'red' zone
Storage Conditions	-40° to 140°F (-40° to 60°C), ambient to 85% Relative Humidity (RH)
Operating Conditions	+59° to 113°F (+15° to 45°C), ambient to 85% RH
Measurement Error	0 – 30 cm H ₂ O = ± 2 cm H ₂ O 30 – 40 cm H ₂ O = ± 3 cm H ₂ O 40 – 60 cm H ₂ O = ± 5 cm H ₂ O

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